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EUCAST News

Clinical breakpoints

Expert rules and intrinsic resistance

Resistance mechanisms

Guidance documents

MIC distributions and ECOFFs

Zone distributions and ECOFFs

AST of bacteria

Media preparation

MIC determination

Disk diffusion methodology

Disk diffusion implementation

Compliance of manufacturers

Breakpoint tables

QC Tables

Calibration and validation

Warnings!

Guidance documents

Projects and data submission

MIC testing services from EUCAST

Previous versions of documents

AST of mycobacteria

AST of fungi

AST of veterinary pathogens

Frequently Asked Questions (FAQ)

Meetings

Presentations and statistics

Documents

Translations

... Warnings!

EUCAST warnings concerning antimicrobial susceptibility testing products or procedures.

The EUCAST disk diffusion development laboratories, a network of laboratories coordinated from the EUCAST development laboratory in Växjö, Sweden, from time to time discover products (disks, media batches, gradient tests or procedures) which are not performing to the expected standard. When this is the case we inform the manufacturer and publish a warning on this page.

We do not systematically test all products so the lack of a warning does not imply that there is no problem with the product in question.

Laboratories which experience problems with a susceptibility test method, and suspect that this may be related to a particular product, may contact EUCAST for advice.

1. Problems with piperacillintazobactam gradient tests from two manufacturers (see below).
2. Wide variation in disk quality in 16 disks from nine manufacturers (see below)

1. Problems with piperacillintazobactam gradient tests from two manufacturers

Following reports of problems related to piperacillin-tazobactam gradient tests, several batches of tests from bioMérieux and Liofilchem (from both single packs and multipacks) were evaluated at the EUCAST Development Laboratory, Växjö, Sweden (January 2016). Enterobacteriaceae and *Pseudomonas aeruginosa* (type strains and strains with resistance mechanisms) and broth micro dilution were used.

Gradient tests from both manufacturers gave variable and unreliable results. The difference between gradient tests and broth micro dilution was not systematic, some values were two dilutions too high, others too low. There was also considerable variation between batches from the same manufacturer.

We urge manufacturers to seriously consider these problems and users to introduce internal quality control of each procedure. We have detected no reason not to trust the disk diffusion (30+6 µg disk) results but currently you can not confirm your disk diffusion results with a gradient test.

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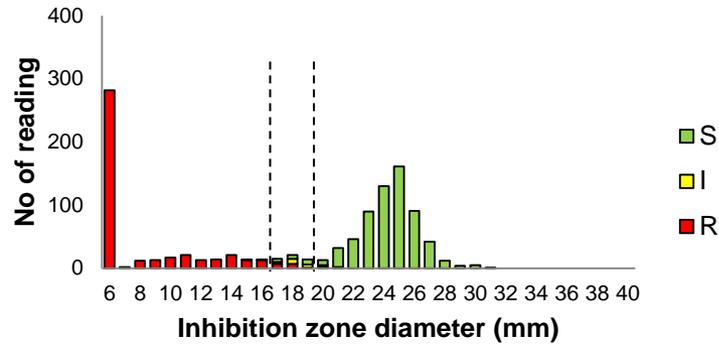
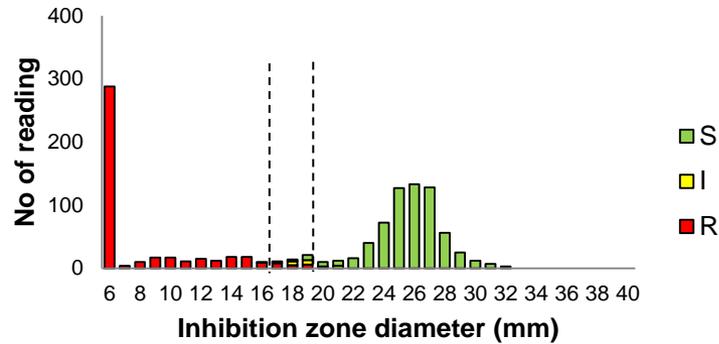
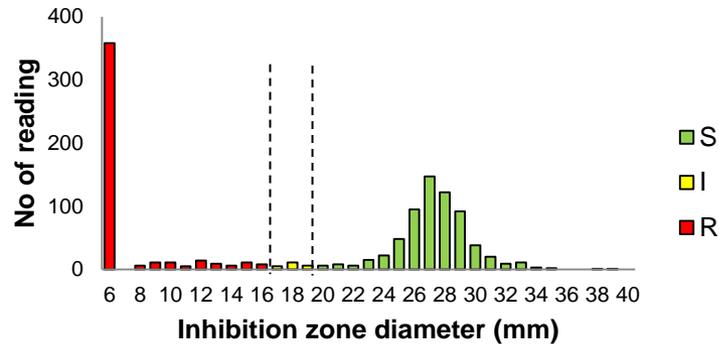
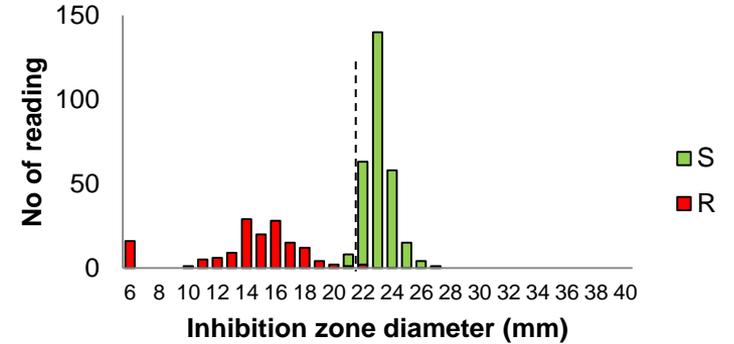
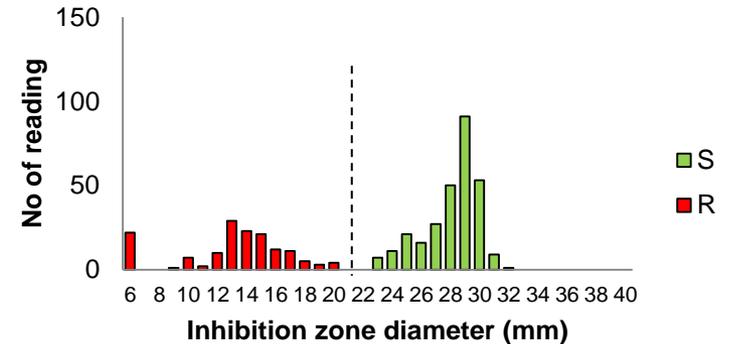
... EUCAST Network Laboratories [▼](#)

EUCAST Network Laboratories

 The EUCAST Network Laboratories form two separate loosely constructed networks of microbiology laboratories with particular expertise and training in EUCAST Antimicrobial Susceptibility Testing (AST) for bacteria and EUCAST Antifungal Susceptibility Testing (AFST) methods, respectively.

EUCAST Network Laboratories are committed to help develop, validate and troubleshoot EUCAST AST/AFST methods and/or to help train and educate other laboratories in EUCAST methods. These laboratories will also play an important role in the development of improved methods and/or assist clinical breakpoint development by providing species-specific MIC datasets.

Not all EUCAST Network laboratories will offer the same services or skills. Some may specialize in educating laboratory staff. Others may be specialized in tests on particular organisms, resistance mechanisms or in methodological development.

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