**miniland**  
BABY

# thermo advanced

Castellano	3	Deutsch	51
English	15	Italiano	63
Português	27	Polski	75
Français	39	Русский	87

[www.minilandbaby.com](http://www.minilandbaby.com)

## Índice

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# Index

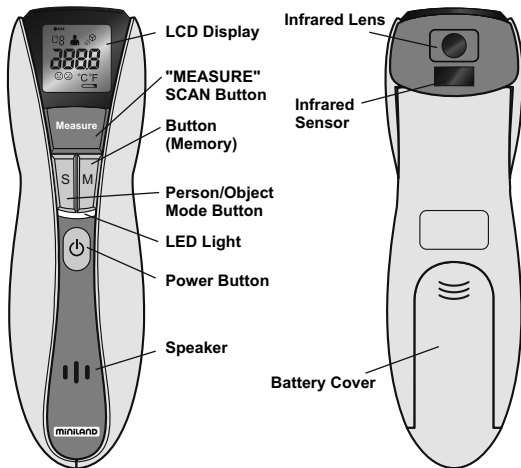
- Introduction
- Product Features
- Infrared Technology
- General Care Information
- Cleaning & Storage
- Helpful Hints
- Changing Modes
- Taking a Reading - Person
- Taking a Reading - Object
- Memory Recall
- Replacing the Battery
- Symbols
- Troubleshooting
- Product Specifications

# Introduction

Thank you for purchasing the Thermo advanced Infrared Thermometer. Please read the following instructions to achieve the most accurate temperatures and safe operation. With proper care and use, your thermometer will provide you with many years of reliable readings. Your Thermo advanced Infrared Thermometer is intended for monitoring human body temperature in the home and is appropriate for all ages. The Thermo advanced can also be used to measure the temperature of objects such as bath water, a bottle of milk or a variety of other items. It's non-contact design makes this thermometer less threatening and more sanitary than most other thermometers. The Thermo advanced can even be used while a child is sleeping.

**NOTE:** Measurements taken with the Thermo advanced Infrared Thermometer are comparable to oral readings.

# Product Features

## Infrared Technology

The Thermo advanced Thermometer features infrared technology that allows for temperature readings to be taken approximately 5 to 8cm from a person or object. By not actually having to touch a person or object to get a reading, the Thermo advanced thermometer offers fast, hygienic results.

## How does it work?

Since all people or objects radiate infrared energy, this thermometer works by changing the infrared energy measurement of a person or object into an accurate temperature reading.

The automatic distance finding sensor ensures the Thermo advanced thermometer is at the ideal distance before a measurement is taken.

In one second, 512 readings are taken. These readings are converted into a single temperature with the peak temperature displayed on the screen.

# General Care Information

Your Thermo advanced Thermometer can provide years of easy and accurate readings if cared for properly.

1. The thermometer should only be used under the supervision of an adult.
2. The infrared technology your Thermo advanced Thermometer uses is completely safe. While it is not harmful to look into the lens while taking a reading, it is not recommended.
3. The Thermo advanced Thermometer is designed to only take human temperature readings from the forehead. Attempting to take a measurement anywhere else on the body may result in an inaccurate reading.
4. Forehead measurements are comparable to oral measurements. Remember temperatures can vary from 36.1°C to 37.2°C (97.0°F to 99.0°F) and still may be considered 'normal'. Time of day, level of activity, exercise and other activities can attribute to variations in temperature. However, unlike oral temperatures, factors such as smoking, talking and drinking do not affect forehead measurements.
5. Do not walk or run while taking a measurement.
6. Clean the lens before and after each use.
7. Store the unit in the protective bag when not in use.
8. Do not touch the lens with your fingers.
9. Do not store the unit where it will be exposed to direct sunlight, dust or humidity. Avoid extreme temperatures.
10. Dropping or subjecting your thermometer to strong shocks should be avoided.
11. Do not attempt to disassemble the unit, except to replace the battery.
12. If measurement is not attainable, the unit must be replaced.

**NOTE:** Performance of the device may be degraded if operated outside of stated temperature and humidity range; stored outside of stated temperature and humidity range; thermometer undergoes mechanical shock (drop).

## Cleaning & Storage

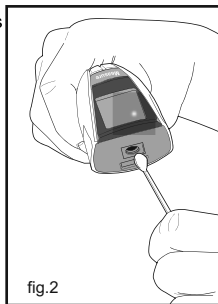
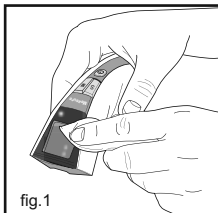
To ensure accurate measurements, clean the lens before and after each use. Fingerprints and dirt can cause inaccurate readings.

### To Clean:

- Use an alcohol swab or a soft cotton swab moistened with alcohol (70% Isopropyl) to clean the thermometer lens, *Figs. 1-2*. Do not use paper towels, the fibers may scratch the lens.
- Do not use water to clean the lens or thermometer.
- Only use a soft dry cloth to clean the thermometer.
- Make sure that liquid does not enter the interior of the thermometer.
- Never use abrasive cleaning agents, thinners or gasoline for cleaning.
- Do not immerse into water or other cleaning liquids.

### To Store:

- Remove the battery if the thermometer will not be used for an extended period of time.
- Store in a sun-free and dust-free dry area at a temperature between  $-25^{\circ}\text{C}$  –  $55^{\circ}\text{C}$  ( $-13^{\circ}\text{F}$  –  $131^{\circ}\text{F}$ ) with relative humidity not exceeding 95%.
- Protect the thermometer from severe impact or shock. Keep stored in the protective storage bag.

## Helpful Hints

- For accurate results, wait at least 30 minutes after exercising, bathing or returning from the outdoors.
- Wait at least 1 minute between readings.
- If multiple readings are taken over a short period of time, use the average of those readings.
- Because everyone's temperature varies slightly, keep a record of your body temperature to serve as a reference when judging a fever.

## Changing Modes

### Switching Between Person or Object

First determine whether you want to measure the temperature of a person or an object.

1. Press the (Power) button to turn on  the thermometer.
2. To switch between the measurement modes for a person or an object, press the 'Mode' button, *Fig. 3*.

The symbol  appears to measure a person and the symbol  appears to measure an object, *Fig. 4*.

### Switching Between Fahrenheit or Celsius

1. Press the 'Power' button to turn the thermometer on.
2. Press and hold both the 'Mode' and 'Memory' buttons for approximately 3 seconds. This will change the mode to either °C or °F, *Fig. 5*. Once the thermometer beeps, the unit is ready for measurement.

**NOTE:** To switch the mode again, wait until the unit beeps and then press and hold both the 'Mode' and 'Memory' buttons until the mode switches to either °C or °F.

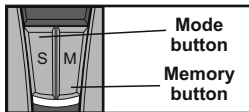
fig.3

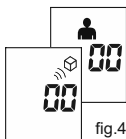
fig.4

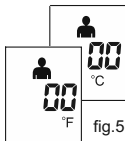
fig.5

## Taking a Reading - Person

**NOTE:** Be sure the thermometer is set to measure the temperature of a person.

1. If the thermometer is off, press the 'Power' button to turn the thermometer on.

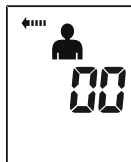
fig.6

2. Press and hold the "Measure" button. The screen should appear like *Fig. 6*.

3. Aim the front of the thermometer at the person's forehead, *Fig. 7*. Hold the thermometer approximately 5 to 8 cm. (2" to 3") away from the forehead.

fig.7

**NOTE:** Attempting to take a measurement anywhere else on the body may result in an inaccurate reading.

4. While continuing to press the "Measure" button, slowly move the thermometer toward or away from their forehead until the thermometer is continuously beeping and the LED light flashes, *Fig. 8*.

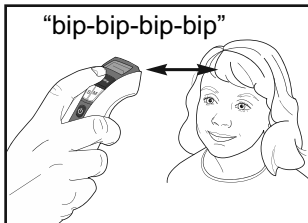
fig.8

5. Once the thermometer is consistently and continuously beeping, release the "Measure" button.

## Taking a Reading - Person

6. The thermometer will beep once and show the temperature reading, *Fig. 9*. The display will illuminate with Nite-Glo® technology for 3 seconds.

**NOTE:** A happy face symbol ☺ will appear on the display if the temperature is below 38°C (100.4°F).

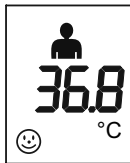
fig.9

If the temperature reading is above 100.4°F (38°F), a sad face symbol ☹ will appear.

If the temperature is outside of the thermometer's sensory range, 'Lo' or 'Hi' will appear on the display. If however, the temperature is outside of the thermometers permissible measurement range, 'Err' will appear on the display.

7. The thermometer will automatically beep and shut-off after approximately one minute of non-use.

**NOTE:** To ensure accuracy:

- Please wait at least 5 seconds between successive readings.
- Clean the probe lens before and after each use.

## Taking a Reading - Object

**NOTE:** Be sure the thermometer is set to measure the temperature of an object.

1. If the thermometer is off, press the 'Power' button to turn the thermometer on.

2. Press and hold the 'Measure' button. The screen should appear like *Fig. 10*.

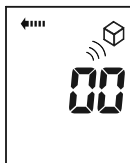
fig.10

3. Aim the front of the thermometer at the object, *Fig. 11*. Hold the thermometer approximately 2" to 3" away from the object and release the 'Measure' button.

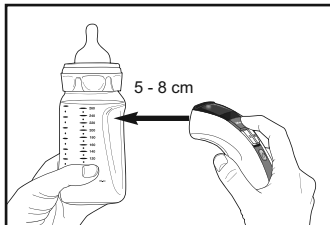
fig.11

4. The thermometer will beep once and show the temperature reading, *Fig. 12*. The display will illuminate with Nite-Glo® technology for 3 seconds.

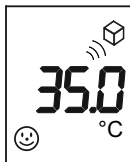
fig.12

5. The thermometer will automatically beep and shut-off after approximately one minute of non-use.

# Memory Recall

## Recalling Measurements in Memory

1. Press the Power button to turn the thermometer on.
2. Press the 'Memory' button to recall up to 10 measurements stored in memory, *Fig. 13*.

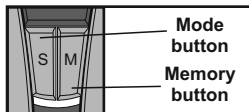
fig.13

3. The  symbol or  symbol will appear with each measurement stored in memory to indicate whether a person or object temperature was taken, *Fig. 14*.
4. The happy face  or sad face  symbol will appear 5 seconds after viewing the measurement stored in memory.

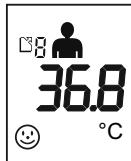
fig.14

**NOTE:** After 10 measurements have been stored in the memory, the oldest reading will be automatically deleted.

## Clearing Memory

1. Press the Power button to turn the thermometer on.
2. To clear the memory, simply hold the 'Memory' button down until the thermometer beeps and two horizontal lines appear, *Fig. 15*.


fig.15


# Replacing the Battery

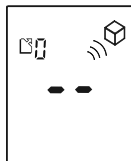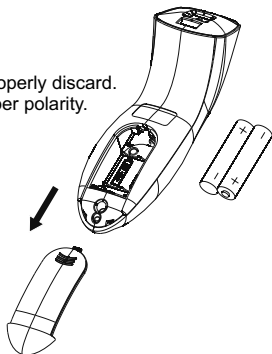When the battery symbol  appears on the screen, the batteries will need to be replaced soon. When the letters "Lo" and the battery symbol  appears on the display, the batteries need to be replaced before taking another measurement. Rechargeable batteries are okay to use in this thermometer.

## Replacing the Battery

1. Gently slide the battery cover back.
2. Carefully remove the old batteries and properly discard.
3. Insert new batteries according to the proper polarity.
4. Slide the battery cover back on.

**NOTE:** Please properly dispose of the batteries away from small children and heat.

## SYMBOLS

Fahrenheit/Celsius	°C/°F
Forehead Scan Mode	
Object Scan Mode	
Memory symbol and measurement number	
Scanning in progress	
Low battery	
Happy face. Indicates temperature is lower than 100.4°F (38.0°C)	
Sad face. Indicates temperature is 100.4°F (38.0°C) or higher	

# Troubleshooting

Display Symbol	Condition/Cause	Corrective Action
	Measurement is below normal temperature range.	Retake temperature ensuring proper positioning.
	Measurement is above normal temperature range.	Retake temperature ensuring proper positioning.
	Operating temperature is outside of the permissible range 16.0°C – 35.0°C (60.8°F – 95.0°F).	Take thermometer to a room where the temperature is within the permissible range. Wait 45 minutes to stabilize thermometer. Retake temperature positioning the probe snugly into the ear canal.
	Blank display	Check battery polarity. May need to replace batteries.
	Low battery	Unit will work; replace batteries soon.
	Low battery warning	Replace the batteries immediately.

# Product Specifications

**Name** Thermo advanced Infrared Thermometer

**Model Number** 89024

**Display System** Liquid Crystal Display

**Accuracy** Human Body:  $\pm 0.3^{\circ}\text{C}$  ( $\pm 0.5^{\circ}\text{F}$ )

**Measuring Range**  $10.0^{\circ}\text{C} - 50.0^{\circ}\text{C}$  ( $50.0^{\circ}\text{F} - 122.0^{\circ}\text{F}$ )

**Memory** Automatically stores up to 10 readings

**Tone** One short beep sounds when the unit is turned ON. Thermometer beeps once for each memory recall and when unit automatically turns off.

**Operation Distance** 5-8 cm (1'97" - 3.14")

**Operating**  $16.0^{\circ}\text{C}$  to  $35.0^{\circ}\text{C}$  ( $60.8^{\circ}\text{F}$  to  $95.0^{\circ}\text{F}$ ) at relative air

**Temperature** humidity (not condensing) of up to 95%

**Storage Temperature**  $-25.0^{\circ}\text{C}$  to  $55.0^{\circ}\text{C}$  ( $-13.0^{\circ}\text{F}$  to  $131.0^{\circ}\text{F}$ ) at relative air humidity (not condensing) of up to 95%

**Automatic Power-off** Automatic shut-off after approximately 1 minute of non-use.

**Battery** 2 - 1.5v alkaline AAA size

**Dimensions** 5-1/4" (L) x 3-3/8" (W) x 1-1/4" (H)  
(134.5mm x 84.8mm x 31.7mm)

**Weight** With battery, 4.27 ounces (121 grams)

**Accessories** Fabric storage, instruction manual and warranty, two AAA batteries

## NORMALIZED SYMBOLS

· Attention: Read the operating Instructions for use!

· Applied part of type BF

· Disposal in accordance with Directive 2002/96/EC (WEEE)

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· Batch code

· Serial number

· The name and the address of the manufacturer

· Authorized Representative in the European Community

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© Miniland, S.A. 2009
**Miniland S.A.** P.Ind. La Marjal I C/ La Patronal 8-10
Babei Sapience Technology Co., Ltd.
Shanghai International Holding Corp.GmbH (Europe)