

BIOFLOW®

Magnetic Boots for Horses and Ponies

Fitting Advice

Bioflow® Horse Boots should be fitted in the same way as conventional brushing boots. Always ensure that the legs to be treated are clean and free of any mud, dirt etc. which may cause a friction sore.

Bioflow® Horse Boots are not recommended for constant use, except in the case of chronic lameness where the horse is stabled and carefully monitored. Treatment for 12 hours out of every 24 is usually sufficient. The Boots should be regularly brushed with a soft brush to remove any surface dirt. They can also be washed in warm soapy water as required.



Important Advice

Bioflow® Horse Boots contain strong magnets; therefore you should avoid contact or close proximity with compasses, watches etc.

Guarantees

The magnetic properties of your Bioflow® Boots are guaranteed for life.

Your Bioflow® Horse Boots are guaranteed against faulty parts and workmanship, subject to normal conditions of use, for a period of one year from the date of purchase.

BIOFLOW®

Email: info@bioflow.co.uk

11/2015