

EPILOG DIAGNOSTIC

Instructions for use

Step 1: create an Epilog Diagnostic account

Go to <https://diagnostic.epilog.care>



Email

Password

LOGIN

Press the “Don’t have an account yet” option.

———— Don't have an account yet?

Forgot your password?

Step 1: create an Epilog Diagnostic account



Fill in your details, view the terms, accept them and submit your information to create your Epilog account.

First name (required)

Last name (required)

Institution

Job Title

Email (required)


I accept the terms of use.

[View terms](#)

SUBMIT

Already a member? [Log in here](#)

Step 1: create an Epilog Diagnostic account

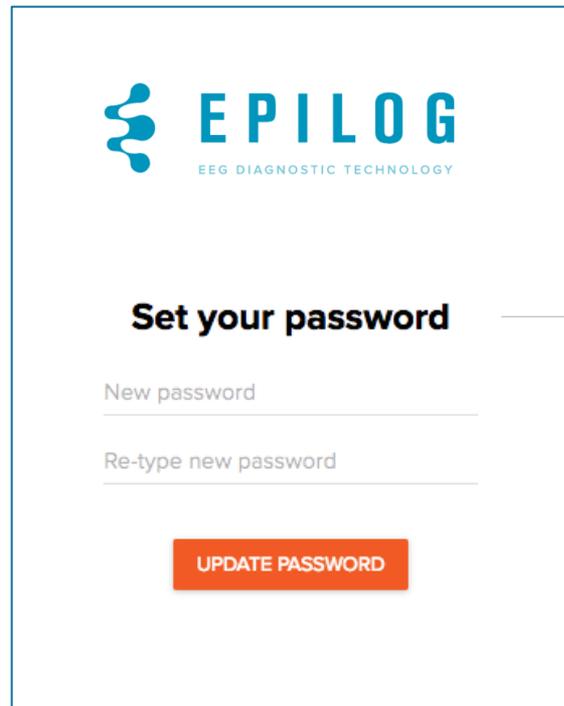

The screenshot shows the Epilog account creation interface. At the top left is the Epilog logo, which consists of a stylized blue icon of three connected circles followed by the text "EPILOG" in a bold, blue, sans-serif font, with "EEG DIAGNOSTIC TECHNOLOGY" in a smaller, lighter blue font underneath. Below the logo, the heading "Set your password" is displayed in a bold, black font. Underneath the heading are two input fields: the first is labeled "New password" and the second is labeled "Re-type new password". Both fields have a light gray border and a horizontal line indicating the input area. At the bottom center of the form is a prominent orange button with the text "UPDATE PASSWORD" in white, uppercase letters.

Check your e-mail and copy paste the link in your web-browser: Choose your password.

Step 2: sign in to your Epilog Diagnostic account



Email

Password

LOGIN

Don't have an account yet?


Forgot your password?

Go to <https://diagnostic.epilog.care> again and fill in your details. Press login.

Add a new patient

Once you signed in to your account, you'll get the overview page of the patients that were submitted for processing, to add a **new patient** you can press the **add new patient** button.

If a new patient is submitted for processing, one slot will be used. This number counts the patients that were submitted for analyses on your account.

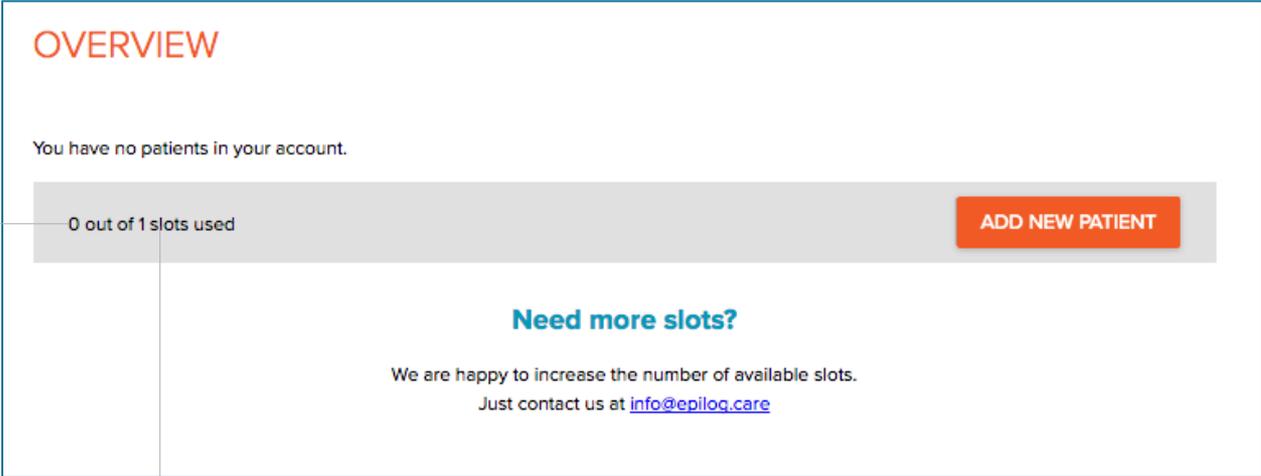


The screenshot shows the 'OVERVIEW' page of the Epilog system. At the top, it says 'OVERVIEW' in orange. Below that, it states 'You have no patients in your account.' A grey progress bar shows '0 out of 1 slots used'. To the right of the bar is an orange button labeled 'ADD NEW PATIENT'. Below the bar, there is a section titled 'Need more slots?' in blue, followed by the text 'We are happy to increase the number of available slots. Just contact us at info@epilog.care'.

The number of slots depicts the number of analyses you still can do on your account. Contact us at info@epilog.care to increase the number of slots. If you have a new account, you will get 1 slot by default.

Add a new patient

Once you press add new patient, you will be asked to give your patient a unique identification code.

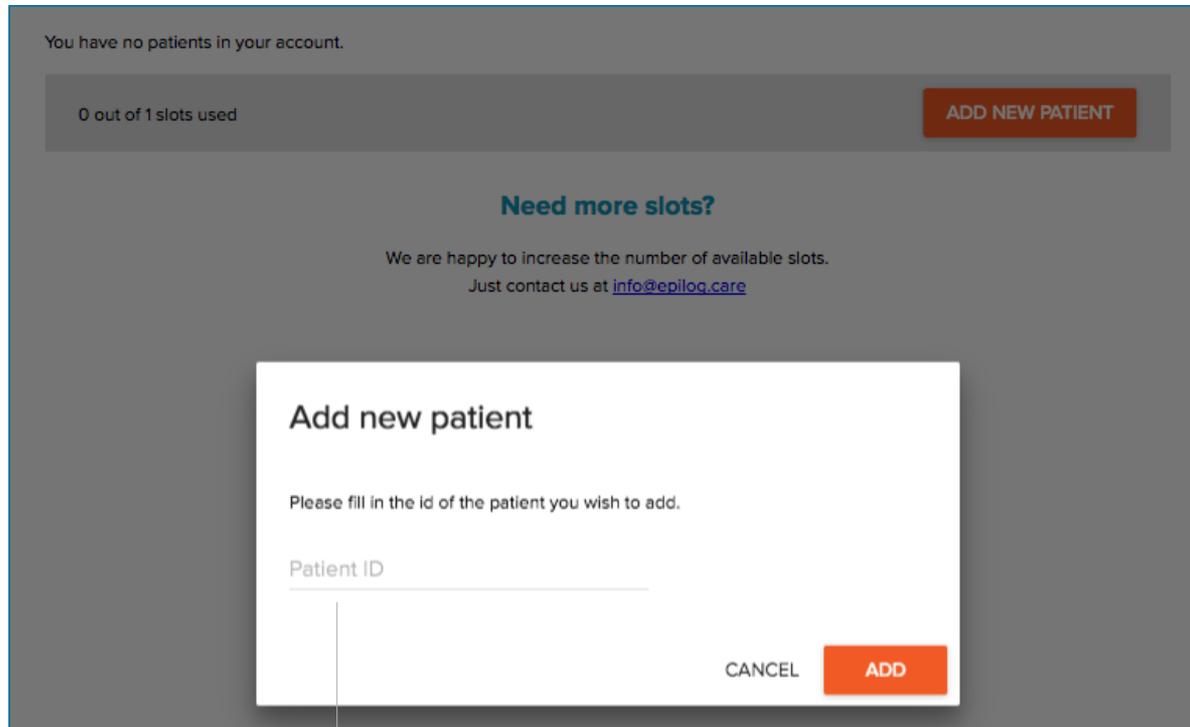

The screenshot displays a user interface for adding a new patient. At the top, it states "You have no patients in your account." Below this, a status bar shows "0 out of 1 slots used" and an "ADD NEW PATIENT" button. A central message asks "Need more slots?" and provides contact information: "We are happy to increase the number of available slots. Just contact us at info@epilog.care". A modal dialog box titled "Add new patient" is open, containing the instruction "Please fill in the id of the patient you wish to add." and a text input field labeled "Patient ID". At the bottom of the dialog are "CANCEL" and "ADD" buttons.

Give the dataset you want to submit a **unique ID**.
You can use this ID to identify your patient.

Add a new patient

After assigning your patient ID, you can select the EEG data. Please read the EEG upload guidelines on the next pages.

The screenshot shows a web interface for editing a patient test. At the top left, it says "EDIT PATIENT TEST" in red, with "Created on Tue Feb 14 2017" below it. In the center, there is a large box titled "EEG" containing an upload dialog. The dialog has the text "DROP FILES OR CLICK BELOW TO ADD" and "CLICK FILENAME TO DELETE" above a cloud icon with an upward arrow. Below the icon is a button labeled "ADD FILES". At the bottom of the main interface, there is a "Notes" section with a text input field and two buttons: "< BACK" and "SUBMIT PATIENT".

Note that the patient ID in this case is TEST

You can drag and drop your files into the EEG upload dialog or press add files to open a file browser.

Add notes for your patient if you want to add an extra specification.

Important!: EEG upload guidelines

We accept EEG data with a **duration more than 6 hours and less than 25 hours** recorded in an epilepsy monitoring unit.

The EEG data must be recorded in one **continuous recording session**. We do not accept multiple recordings of the same patient in one submission.

We accept EEG data from most of the hardware vendors - please provide us the raw EEG data files.

We accept EEG recordings in a **standard clinical setup** (according to the **international 10-10 or 10-20** positioning system).

The **number of electrodes** should be **higher or equal than 17**.

If you have multiple files from the same recording, please put the files in a folder first and upload them in a zipped folder.

Step 3: submit your patient data for Epilog Diagnostic analysis

EDIT PATIENT TEST

Created on Tue Feb 14 2017

EEG

DROP FILES OR CLICK BELOW TO ADD
CLICK FILENAME TO DELETE

✓ MRI.zip



ADD FILES

Notes

← BACK

SUBMIT PATIENT

After uploading the EEG files, the submit button will become active. Submit the files for processing.

Patient status is submitted


test

Thu Nov 09 2017

SUBMITTED...

After the patient data is submitted it will get the **status submitted**. Once Epilog starts processing the data, the status will change from **waiting** to **processing**. The **results** will be **ready within 24 hours**.

Access and interpret the results

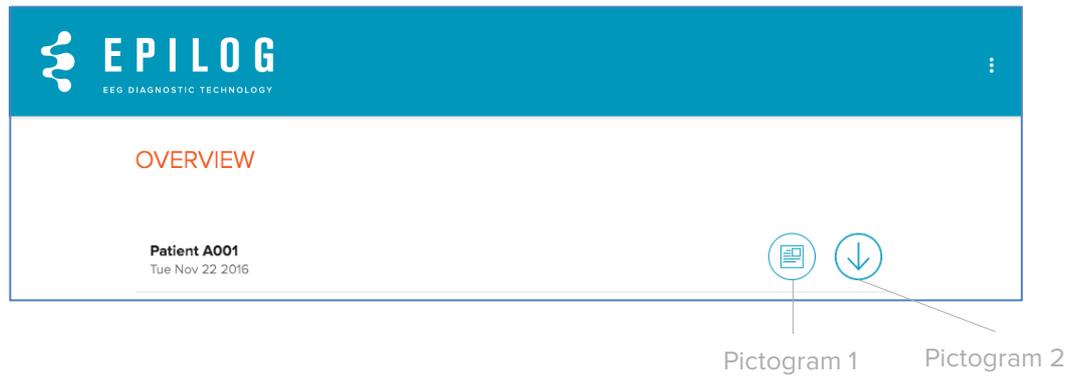

Once the **results** of the patient are ready, the status changes into **two pictograms**. The **first** corresponds with the **pdf report**. Please read the [Epilog diagnostic report interpretation guidelines](#) for more information. The second pictogram corresponds with a download option.

The Epilog Diagnostic software is for professional use only



Requires prescription in the United States.



Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.



Indicates the need for the user to consult the instructions for use.

© Epilog NV, All rights reserved

Address: Vlasgaardstraat 52

9000 Ghent, Belgium

Product: Epilog Diagnostic v1