
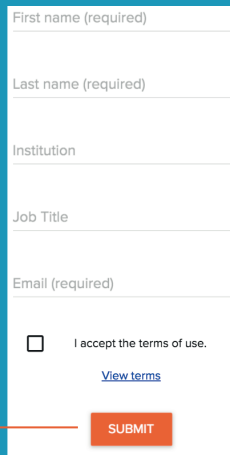


# QUICK - GUIDE EPILOG DIAGNOSTIC


1 Go to **diagnostic.epilog.care** :

Click on “**don’t have an account yet**”

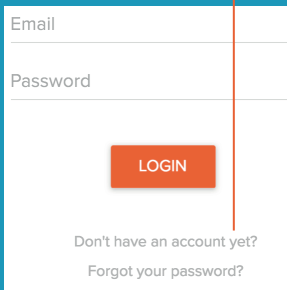

Fill in your details and click **SUBMIT**  
Epilog will send you an e-mail

2 Click on the link in the e-mail you received

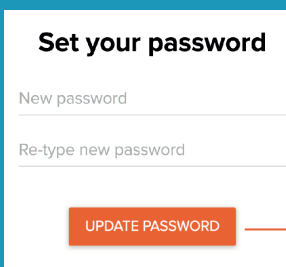


Choose your password  
and click on **UPDATE PASSWORD**


You are now ready to use EPILOG **DIAGNOSTIC** for your patients

3 Go to **diagnostic.epilog.care**,  
enter your e-mail and password

Click **LOGIN**

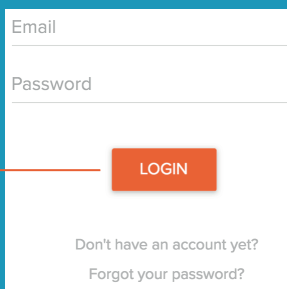

4 Click **ADD NEW PATIENT**

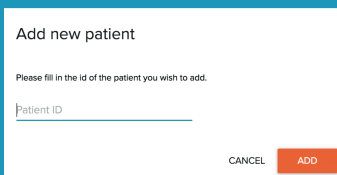

Choose the patient’s ID  
and click **ADD**

5 Drag and drop the EEG files and press **SUBMIT PATIENT**

Within 24 hours the results are ready  
and you will be notified by e-mail.

Follow the link in the e-mail and login  
to interpret the results.

