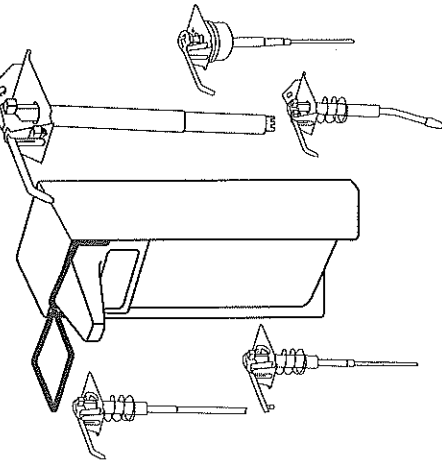
ingo-man® 26

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Dispenser for soap and disinfectant
with interchangeable pump units

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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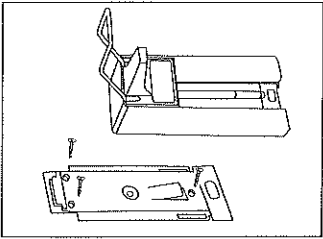
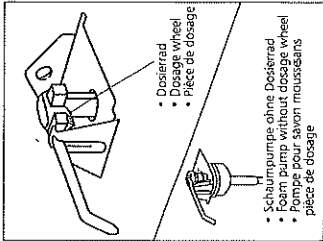
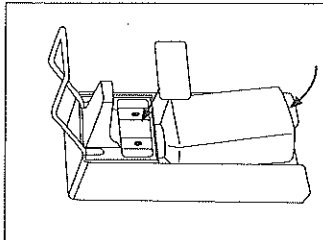
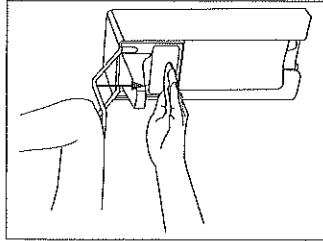
Check before installation

1. The dispenser should be fitted above a sink so that wet hands can drip.
2. Required space above the dispenser is 40 mm; for below refer to dimensions overleaf.
3. Required height: 130–135 cm, from floor to top of dispenser.

Mounting

Remove the container. Slightly press the fixing catch and pull down the wall plate. Fix wall plate with enclosed plugs and screws. Mount the dispenser onto the wall plate by firmly pressing until it clicks into place.

Dosage

The dispenser is adjusted to the maximum amount of approx. 1.5 ml/stroke. If required, the dosage can be reduced to a minimum of approx. 0.5 ml/stroke by gradually turning the dosage wheel head. The foam pump has not a dosage wheel. The dosage is not changeable.

Container exchange

Insert the refilled container from below and press the facial clip into position.

Operation

For initial working, the unit must be operated several times until it delivers fully. Hold the open hand under the outlet nozzle.

Cleaning/Sterilizing

Press locking device and push dispenser upwards. Dispensers using aluminium or stainless steel for units fitted with a stainless steel pump or stainless steel pumps alone (also the pump type 28 with flexible sucking tube) are autoclavable up to 121 °C, 1 Bar. Plastic pumps and powder coated dispensers are not autoclavable!

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