



Decontamination Procedures:

1. Hand Wash

The device is cleaned down with a damp cloth using mild detergent and/or disinfectant. For example, a mild chloride disinfectant (solution of 1000 parts per million) can be used. Excess water is then removed from the device using a towel or a compressed air tool. The device is then placed in a suitable environment to fully dry.

2. Thermal Decontamination

Components (for example, ferrules and tube ends) may be removed to prevent water collection. The device is placed in an aqueous system (for example, a Turbex machine) for thermal disinfection processing at 80°C. After the process, any excess water should be removed from the device using a towel or a compressed air tool and the device should be checked for any pools of water. The device is then placed in a suitable environment to fully dry.

3. Chemical Decontamination

Components (for example, ferrules and tube ends) may be removed to prevent water collection. The device is placed in an aqueous system (for example, a Turbex machine) for chemical disinfection processing at 60°C. After the process, any excess water should be removed from the device using a towel or a compressed air tool and the device should be checked for any pools of water. The device is then placed in a suitable environment to fully dry.

4. Industrial Washing Machine

The device or components (mainly fabric items) are washed in an industrial spin washer. Unless the components are suitable for use in an industrial dryer, then excess water should be removed from the device using a towel or a compressed air tool and then the device placed in a suitable environment to fully dry.

5. Industrial Dryer

The load may be then dried in an industrial dryer.

6. Autoclave

The device is processed in an Autoclave for 3 minutes up to 137°C

Product Groups	Hand Wash	Thermal Decom.	Chemical Decom.	Industrial Washer	Industrial Dryer	Auto-clave
Wheelchairs						
Powerchairs						
Walking Canes						
Walking Frames						
Rollators						
Triwalkers						
Handy Grabbers						
Footstools						
Handy Trolley						
Overbed Tables						
Perching Stools - PVC						
Perching Stools - PU						
Bathlift – Riviera / Bellavita						
Bath Boards						
Slatted Bathing						
Shower Stools						
Raised Toilet Seats						
Toilet Frames						
Stacking Commodes						
Glideabout Commodes						
Community Beds						
Bed Sticks and Side Rails						
Otter – Paediatric						
Nimbo – Paediatric						
Armchairs – PVC						
Armchairs – Leather						
Armchairs – Fabric						
Mobility Scooters						