



Back Up Battery Installation Guide

Many of the Restwell Rise and Recline armchairs come with a battery operated reset facility. This will allow the user to install disposable batteries so that the chair will lift or return to its normal position in the event of a power cut. Batteries should be replaced annually or when the reset facility has been used once.

Back Up Battery Installation – Chairs with Separate Power Supplies

Applies to chairs: Boston from 2013 (CLR1Y and CLR3Y), Borg (BRC001 and BRC002), Oakland (CLR23), Dakota (ILR12Y), Intalift(ILR4). Batteries Required: 2x 9V PP3.



1. Remove the battery compartment lid from the power supply by pushing the lid away from the power supply.



2. Pull out the two battery connectors from the compartment in the power supply and push a 9V battery on to each of the connectors.



3. Place the batteries and wiring inside the battery compartment of the power supply.



4. Slide the battery compartment lid back on to the power supply.

Back Up Battery Installation – Chairs with Control Boxes

Applies to: Nevada and Portland (CLR12 and CLR22), Memphis (CLR12), Denver (CDL8), Boston (prior to 2013), Chicago (Prior to 2011). Requires 2x 9V PP3 batteries:




1. Raise the chair and lift up the fabric access flap at the rear to locate the control box. Using your finger nail, lift the locking tab on the bottom of the battery compartment lid and release.



2. Reach in to the battery compartment and pull out the two battery tab terminals from the compartment.



3. Connect two 9V batteries to the tab terminals pulled out from the battery compartment.

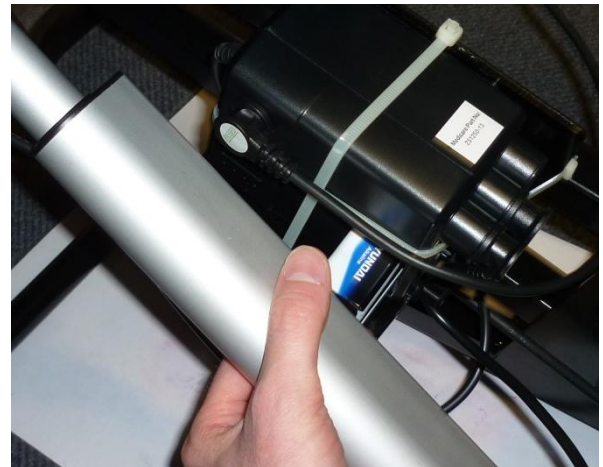

4. Push the two batteries back in to the compartment and reattach the battery compartment lid.

© Drive Medical Ltd
Ainley's Industrial Estate,
Elland, West Yorkshire, UK HX5 9JP.

Issued 07/14