

# COMFY DIAMOND MASSAGE ARMCHAIRS OWNERS HANDBOOK

## CONTENTS

1. Introduction
2. Parts Description
3. Personal Safety
4. Installation Instructions
5. Operating the Armchair
6. Massage Function
7. Care and Maintenance
8. Specification
9. Fair Wear and Tear Policy
10. Warranty

## INTRODUCTION

The Comfy Diamond range of massage armchairs are designed to be comfortable, stylish whilst offering excellent value for money.

The armchairs come in a variety of fabrics and styles to suit most decors, and are made to the highest quality and are CE marked. The weight limit for the armchair is 21 stone (136kg).

Comfy is a brand of Medicare Technology. Medicare Technology is an ISO 9001 certified company. For more details about our products please see your local dealer or contact:


**Medicare Technology Ltd,**  
Dale House, Armytage Road, Brighouse,  
West Yorkshire HD6 1PT.

Tel: +44 1484 727301 Email: [enquiries@medicaretechnology.com](mailto:enquiries@medicaretechnology.com)

Web: [www.medicaretechnology.com](http://www.medicaretechnology.com)



## PARTS DESCRIPTION

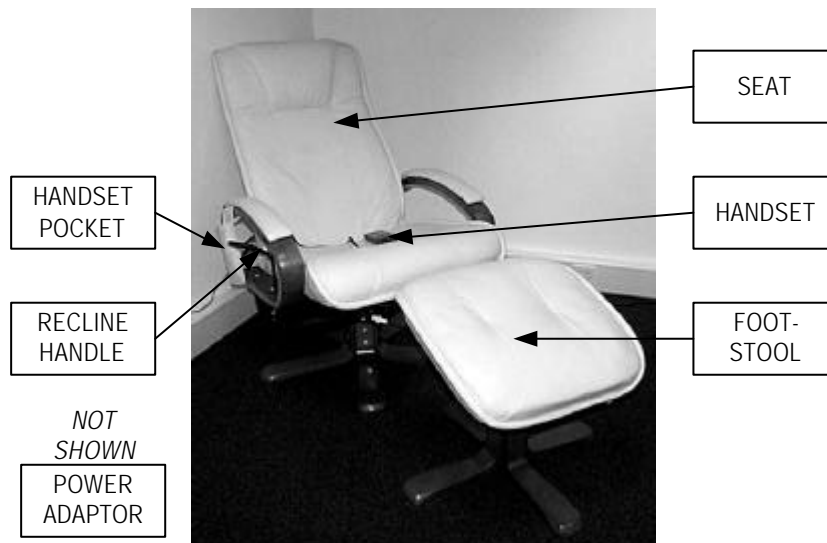

## PERSONAL SAFETY

- Ensure that only Medicare parts are used on the armchair. Periodically visually inspect the handset and power cord for signs of wear, chafing or cracks.
- When reclining the armchair, ensure that you will not collide with anybody behind you. Please ensure that no children or pets are underneath the armchair before operating the recline mechanism.
- Carelessness Causes Fire. Caution should be taken to avoid fire. Please ensure that all cigarettes and matches are properly extinguished. Please be careful of other fire hazards such electric and gas fires, etc nearby.
- The weight limit on the armchair is 21 stone (136 kg). Please ensure that only one person uses the massage chair at any one time.


# INSTALLATION INSTRUCTIONS

1. Unpack the armchair and footstool and place them in your desired position. Be careful not to put the back of the armchair too close to a wall. Ensure that the chair can fully recline without colliding with any walls or objects behind. (See Operating the Armchair about reclining).
2. Connect footstool to the armchair using the two round electrical connectors on the footstool and underneath of the armchair.
3. If necessary, connect the handset to the armchair by plugging the male 8-pin plug on the handset to the 8-pin female plug on the armchair.
4. Finally, plug the power supply in to the armchair using the round DC power plug and plug the power supply in to the mains. Switch on the mains power supply.

# OPERATING THE ARMCHAIR

1. To recline the armchair first sit in the armchair. Then twist the recline handle and push back on the back rest of the armchair. Release the handle and push on the back rest until it locks in to the desired position.
2. To raise the armchair whilst not in the armchair, twist the handle whilst pushing the back of the armchair forward. Release the handle and push on the back rest until it locks in to the desired position.
3. To raise the armchair whilst sat in the armchair, twist the handle whilst leaning forward in the armchair. Release the handle and lean forward until the chair locks in to the desired position.

# MESSAGE FUNCTION

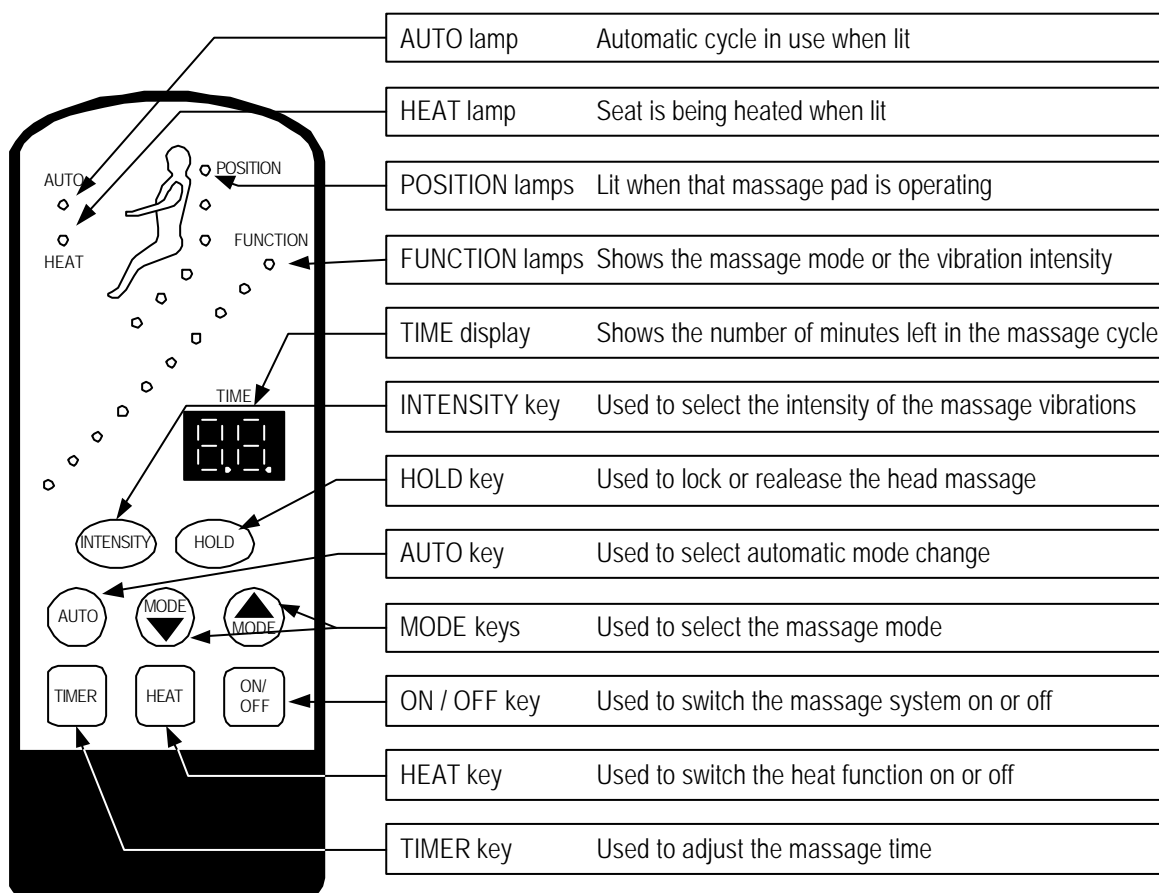

# MESSAGE FUNCTION

## To operate the Massage function:

- Press the ON / OFF key to switch the massage unit on. The massage unit will start with a 30 minute massage programme which automatically alters the massage mode every minute. Press the ON / OFF key again to switch off the unit.
- Press the HEAT key to switch the heat function on. The HEAT lamp will be lit when this function is on. Switch the HEAT key again to switch the heat function off.
- To adjust the time which the massage unit will run for (shown by the TIME display) press the TIMER key. The time will then increase up to a maximum of 99 minutes, after which pressing the TIMER key will reset the time to 1 minute.
- Press the AUTO key to switch the automatic mode cycle function off, which stops the unit from changing massage mode every minute. Press the AUTO key to switch on the automatic mode cycle function. The AUTO lamp is lit when the automatic mode cycle function is on.
- Press one of the MODE keys to move up or move down a massage mode. The current massage mode is shown by one of the ten FUNCTION lamps being illuminated.
- Press the INTENSITY key to increase the intensity of the massage vibrations. After pressing the key, the new intensity will be shown momentarily by illuminating a number of the FUNCTION lamps (the number lit depends on the intensity). Once the intensity is at maximum, pressing the key again will reset the intensity to it's lowest level.
- Press the HOLD key to lock the head massage in it's current state. Press the key again to release the lock.

# CARE AND MAINTENANCE

- When not in use, switch the massage unit off and disconnect from the mains.
- Do not rub the surface of the chair with hard or coarse objects.
- Keep the chair dry and clean.
- Clean the upholstered parts of the chair with a dry cloth. The wooden parts of the chair may be cleaned with a duster or dry cloth. Polish should not be used as this may cause abrasion of the wood.

# SPECIFICATION

Two styles of massage armchairs complete with footstool. Upholstered in a choice of fabrics with matching polished wood arms and legs. Available upholstered in cream leather, black leather, black pvc, amber velour, red velour or blue velour.

The seat on the armchair measures 530mm wide x 510mm deep.

The massage unit uses a Power Supply Unit which outputs 12V 1200ma (14.4VA).

Dimensions (millimetres)	Armchair			Footstool		
	Length	Width	Height	Length	Width	Height
MC1	730	720	1020	460	430	450
MC2	750	720	1080	460	430	480

# FAIR WEAR AND TEAR POLICY

Your new armchair is covered by the warranty below. Please note however this warranty does not include:

- *Damage from misuse, and use over the specified weight limit.*
- *Material fading over time or through sunlight.*
- *Damage from spills or leaks.*
- *Collision damage (rubbing against walls, etc)*
- *Accidental tears and damage*
- *Cigarette and match burns*

## WARRANTY

Your Comfy Diamond massage armchair comes with a comprehensive twelve-month warranty. For warranty issues and claims please contact your Medicare Technology dealer.

### Warranty Conditions:

1. Any work or replacement part installation must be carried out by an authorised Medicare Technology dealer / service agent.
2. To apply the warranty should your armchair require attention please contact the your Medicare Technology dealer / service agent.
3. Should any part of the armchair require repair or full or part replacement as a result of a manufacturing or material defect within twelve months of receiving the armchair, parts will be supplied free of charge.  
Note: The guarantee is not transferable
4. Any repaired or replaced parts will be covered by this warranty for the balance of the warranty period on the armchair.
5. Parts replaced after the original warranty has expired will be covered by a three-month warranty.
6. Consumable items supplied will not generally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or material defect.  
Such items include (among others): upholstery, batteries
7. The above warranty conditions apply to brand new chairs. If you are unsure whether your chair is covered, check with the service agent/dealer.
8. Under normal circumstances, no responsibility will be accepted where the armchair has required assistance as a direct result of:
  - a) The armchair part not having been maintained in accordance with the manufacturer's recommendations.
  - b) Failure to use the manufacturer's specified parts
  - c) The armchair or part having been damaged due to neglect, accident or improper use
  - d) The armchair or part having been altered from the manufacturer's specifications or repairs having been attempted before the service agent / dealer is notified.

In the event of your armchair requiring attention, please contact your service agent / dealer and give all relevant details so they can act quickly. Please contact Medicare Technology for advice if you do not know where the armchair was originally purchased from.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.