



KARRISTICK OWNER'S MANUAL

1. CONTENTS

- | | |
|--------------------|------------------|
| 1. Contents | 4. Operation |
| 2. Introduction | 5. Specification |
| 3. Personal Safety | 6. Warranty |

2. INTRODUCTION

The Healthtech Karristick is designed as a shopping aid and to help you feel more stable and secure when walking. The aid is constructed from lightweight aluminium tubing with two wheels and height adjustable push handle. Healthtech is a brand of Medicare Technology and offers a variety of walking and mobility aids to suit most purposes.

Medicare Technology is an ISO 9001 accredited manufacturer and all devices are produced to the highest standard.

3. PERSONAL SAFETY

- The Karristick is designed primarily as a shopping aid. It should not be used as a mobility aid (such as a walking stick). For details about mobility aids such as walking sticks please contact your Medicare Technology dealer.
- Be careful not to overload the Karristick with too much shopping, or with more shopping than you feel comfortable with.
- Ensure that the weight of any shopping carried is equally distributed between both sides of the Karristick.
- Do not put the whole of your weight on the Karristick.
- Always fully fold out the Karristick before use. Also check that the height adjustable handle is locked in position before use.

4. OPERATION

- Fold out the Karristick before use. Fold out the Karristick by pushing the front and rear wheels apart. Then push down on the front of the sliding cross brace to ensure the Karristick is fully folded out.
- The height of the handle on the Karristick can be altered. To alter the height of the handle, push in the two brass buttons and then adjust the height of the handle until the buttons lock in the desired position.
- The Karristick can be folded up to aid storage. To fold up, pull up the front sliding cross brace until the Karristick is locked in the storage position.
- Operate the Karristick by wheeling it along the ground using the handle. To turn with the Karristick turn the handle in the direction of travel.
- The Karristick has two hooks to carry shopping bags. These are located below the handle.
- The Karristick can stand up unattended when folded out. Please ensure if you do this, that you balance the weight of the shopping equally between both sides of the Karristick.

5. SPECIFICATION

- Tubular aluminium construction with four wheels, height adjustable handle, reflector and two carry hooks
- Weight 1kg (2.2lb)
- Maximum dimensions (cm/in):
- Folded (W x D x H): 23 x 10 x 89cm / 9" x 4" x 35"
- Operating (W x D x H): 52 x 10 x 81cm / 20½" x 4" x 32"
- Supplied by:

Medicare Technology Limited

material defect. Such items include (among others): upholstery and tyres.

7. The above warranty conditions apply to brand new Karristicks. Second-hand Karristicks supplied directly by Medicare Technology carry a six-month warranty period. If you are unsure whether your Karristick is covered contact your dealer.
8. Under normal circumstances, no responsibility will be accepted where the Karristick has required assistance as a direct result of:
 - a. the Karristick part not having been maintained in accordance with the manufacturers recommendations
 - b. failure to use the manufacturer's specified parts
 - c. the Karristick or part having been damaged due to neglect, accident or improper use
 - d. the Karristick or part having been altered from the manufacturer's specification or repairs having been attempted before the dealer is notified

In the event of your Karristick requiring attention, please contact your service agent / dealer and give all relevant details so they can act quickly.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

6. WARRANTY AND MAINTENANCE

To improve the safety and lifespan of your Karristick, it is recommended that you have it serviced on a regular basis. Recommended Service Schedules for Karristicks are available to service agents.

There is a comprehensive twelve-month warranty from the date on which your new Karristick is delivered. The warranty covers the Karristick for repairs or replacement during this period. For more detail, please see the warranty conditions overleaf:

1. Any work or replacement part installation must be carried out by an authorised Medicare Technology dealer / service agent.
2. To apply the warranty should your Karristick require attention please contact the outlet from which you purchased the Karristick.
3. Should any part of the Karristick require repair or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. Note: The guarantee is not transferable.
4. Any repaired or replaced parts will be covered by the balance of the warranty period on the Karristick.
5. Parts replaced after the original warranty has expired will be covered by a three-month warranty.
6. Consumable items supplied will not generally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or

Other Products in the Medicare Technology Range include:



Please contact your local Medicare Technology Dealer for a catalogue.

Medicare Technology Ltd,
Dale House, Armytage Road,
Brighouse, West Yorkshire HD6 1PT.
Tel: 01484 727301
www.medicare-technology.co.uk

Dealer Stamp