




Note: Remove all packaging before installation

	<p>1. Handcontrol Installation. Unscrew the 2 short bolts from the back of the handcontrol using the allen key provided. Attach the handcontrol to the handgrip with the extra section by reattaching the 2 short bolts.</p> <p><i>Note: For right-hand use the handcontrol should be mounted to the LEFT of the handgrip. For left-hand use the handcontrol should be mounted to the RIGHT of the handgrip.</i></p>
	<p>2. Handgrip Installation. Remove the existing wheelchair handgrips. Remove the rubber end from the PowerStroll handgrips and loosen the bolt inside the handgrip using the allen key provided. Slide the new handgrip on to the wheelchair push handle and retighten the bolt. Tighten up the 2 grub screws on the handgrip to secure the handgrip. Repeat for the other handgrip.</p> <p><i>Note: Do not overtighten the grub screws as this could damage the handgrip.</i></p>
	<p>3. Bracket Installation. Unscrew both halves of the brackets and put together the 2 sections of the spring-loaded bar. Insert the halves of the brackets with protruding pegs in to the spring-loaded bar. Place the bar where it is required on the wheelchair (the spring in the bar will keep the brackets in position during installation). Attach the outer halves of the brackets using the threaded knobs provided. Remove the bar.</p>



4. **Powerpack Installation.** Insert the spring loaded bar in to the bracket at the front of the powerpack. Install the spring-loaded bar with powerpack on to the 2 brackets now mounted to the wheelchair. Position the powerpack centrally and secure by tightening the 2 threaded knobs. Connect the powerpack to the handcontrol using the coiled cable from the powerpack.



5. **Install the battery.** Place the battery on the tray on top of the powerpack. Connect the battery to the powerpack.

*Note: Read the Owner's Handbook fully before using the PowerStroll
If you have any queries regarding the PowerStroll,
please contact your Medicare Technology outlet.*