

*Note: Remove all packaging before installation*



### 1. Handset Installation

Loosen both knobs on the handset.

Squeeze the crocodile clip on the handset and apply to the push handle as shown. Secure the clip by turning the outer knob clockwise.

Tilt the handset to the required angle then tighten the inner knob clockwise to secure.



### 2. Bracket Installation

Install both brackets on the bottom horizontal tubing of the wheelchair, using an Allen Key.

Ensure that the screws are sufficiently tight and do not rotate on the tubing.

Attach the two sections of Velcro together.



### 3. Powerpack Installation

Feed the telescopic bar through the front end of the powerpack. Loosen the tristar knob on the bar and adjust to the correct length.

Tighten the knobs on the bar and at the front of the powerpack. Place the two ends of the bar on the rear of the brackets, then push the powerpack forward until it locks in position.



#### 4. Battery Pack Installation

Place the battery pack in to the battery tray as shown in the photograph.

Secure the battery pack using the two Velcro strips on the side of the plastic battery tray.



#### 5. Final Connection

Connect the cable from the powerpack in to the underneath of the handset.

Place the top loop of the lift-up strap around the opposite push handle to the handset.

*Note: Read the Owner's Handbook fully before using the PowerStroll*  
*If you have any queries regarding the PowerStroll,*  
*please contact your Drive Medical outlet.*

*Note: Wheelchairs fitted with attendant-controlled cable brakes.*

*On such wheelchairs, the attendant brake levers should be removed prior to fitting. If the attendant brakes do not have a separate parking brake facility, then these should be removed and parking brakes added.*