

# Lightweight Walking Frame




The lightweight walking frame is suitable for indoor and outdoor use. It is made from lightweight aluminium tubing and is foldable for portability and storage. It features height adjustable legs with rubber ferrule tips for comfort and security. It is suitable for users up to 28 stone (180kg).

The walking frame comes with two comfortable handles to grip when using the walking frame.


## Height Adjustment

The height of the legs on the walking frame can be adjusted by up to 18cm (7"). To adjust leg height, first push in the brass button on leg (see photograph). Adjust the height of the leg until it is locked in position and the brass button can be seen again.

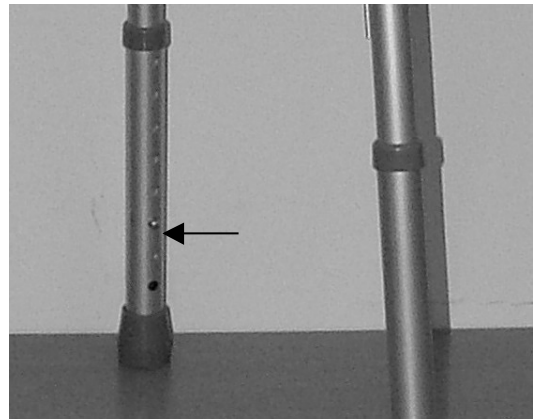

## Folding the Walking Frame

Always fully fold out the walking frame before use. To do this, pull the side away from the front of the frame, until the side locks in position (as shown by the white arrow in the photograph). Repeat for the other side.

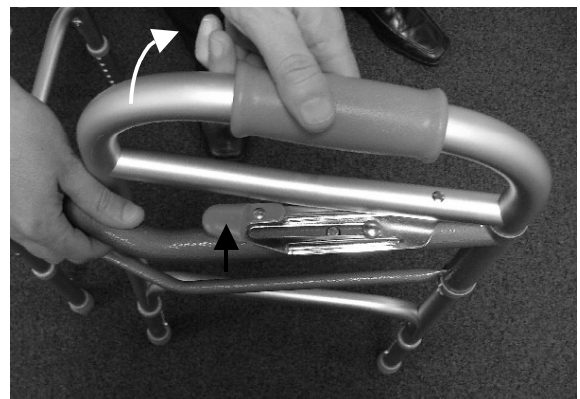

To fold up the walking frame for storage, push the button (shown by black arrow in photograph) whilst pushing the side back towards the front of the walking frame.

## Personal Safety

- For your own personal safety and comfort, adopt a comfortable posture where you can use the walking frame safely and securely.
- Store the unit in a safe environment.
- Ensure the walking frame is fully folded out and locked in position before using.
- Ensure that all the legs on the walking frame are set at the same height, and all 4 legs are on the ground before using.

## Care and Maintenance

- Ensure the walking frame is only used as a mobility aid.
- Ensure the walking frame is kept clean. Wipe down the walking frame with a damp cloth on a regular basis.

## Specification

- Dimensions (W x D):  
56 x 46cm / 22½” x 18”
- Height: 81 – 99cm / 32” – 39”
- Weight 3kg / 6.6lb
- Maximum user mass 180kg / 28 stone

## Warranty

The Walking Frame comes with a twelve-month warranty from date of purchase. The warranty covers design and manufacturing defects. It does not cover accidental damage or wear and tear. For full details of the warranty please contact your dealer.

Manufactured by:

**Medicare Technology Limited**



**E: [enquiries@medicaretechnology.com](mailto:enquiries@medicaretechnology.com)**

**W: [www.medicaretechnology.com](http://www.medicaretechnology.com)**

