



Padded Cane Seat (WS012)

User Instructions

The cane seat can be used as a temporary seat or as an aid to walking.

The cane seat is constructed from lightweight aluminium tubing with padded vinyl seat and features a height adjustable handle. The cane seat can be cleaned using a damp cloth and mild detergent.

The cane seat is suitable for users up to 16 stone (100kg).

Instructions for Use

- To use the cane seat as a seat, push the bottom of the seat away from the main frame and fold out the seat.
- To use the cane seat when walking, fold up the seat ensuring the seat locks shut. Then walk holding the cane seat by the foam grip handle.
- To alter the height of the handle, first pull out the 'E' clip. Then reposition the handle as required and push 'E' clip back in to secure.

Precautions

- Do not exceed the maximum weight capacity of 16 stone (100kg).
- The cane seat should only be used by one person at a time.
- Users with mobility problems should not use the cane seat as a load bearing walking stick.
- Ensure that the cane seat is placed on level ground with a suitable surface before using as a seat.

Manufactured by:

Medicare Technology Limited

Wheeltech

Comfy

Heartech

Ainleys Industrial Estate, Elland, West Yorkshire HX5 9JP.
 T: 01422 314488 W: www.medicaretechnology.com
 E: enquiries@medicaretechnology.com