




# Locking Clamp for Raised Toilet Seats Installation Instructions

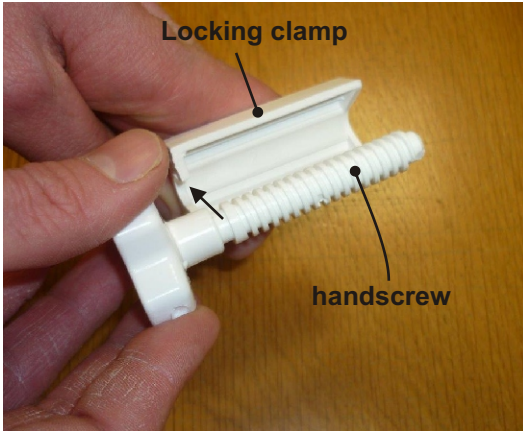



## Step 1

Place the collar of the Handscrew in to the semi-circle cut-out on the Locking Clamp

## Step 2

Slide the Locking Clamp in to the void in the Raised Toilet Seat.

Turn the Handscrew clockwise for two rotations to attach the Locking Clamp

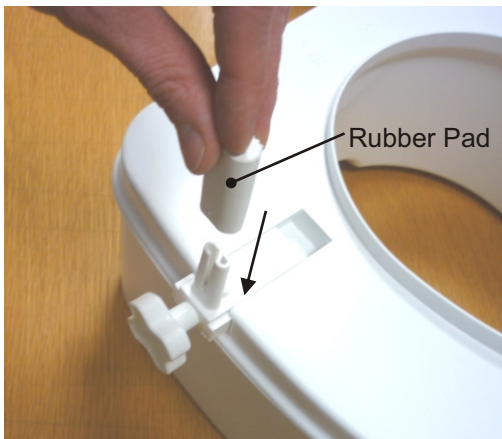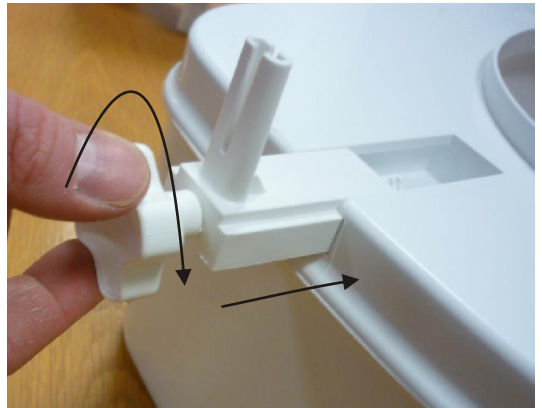

## Step 3

Place the Rubber Pad over the prong on the Locking Clamp with the ridged side facing inwards.

Repeat Steps 1 to 3 for the other Locking Bracket and then follow the Raised Toilet Seat Instruction to install the Seat on the toilet.