

warranty details

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

drive

Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB Hx5 9JP
www.drivemedical.co.uk

Document code: SBS_Hangtag (Issued Aug 2011)

slatted bath seat

drive

CE

www.drivemedical.co.uk

installation and removal

Place the bath seat in the bath in the desired position.

When removing the bath seat, first pull the tabs on the suckers to release them then lift the seat out of the bath.

using the bath seat

Ensure you read the Safety section before using the bath seat.

The bath seat is available with 6", 8" or 12" height and has 5 seat slats.

They are designed for a user to sit on and enable them to bathe, wash or shower in comfort. They have a weight limit on 190kg (30 stone).

Consult your healthcare professional for advice about using the product safely.

cleaning the bath seat

Remove the seat from the bath before cleaning

All parts of the bath seat should be cleaned periodically using normal household detergents.

When disinfecting is required, a mild disinfectant that will not cause damage to plastics should only be used.

Strong acid, alkaline or cleaning products may cause damage or discolouration to the product.

After cleaning or disinfecting, the parts should be showered down or wiped with a damp cloth before the bath seat is reinstalled.

Autoclave cleaning is not recommended as it can cause damage to the specialist plastics used in the bath seat's construction.

bath seat safety

- The bath seat should only be used as an aid for sitting. Do not stand on or misuse the bath seat.
- Do not exceed the weight limit of the seat.
- Some users and children may require supervision whilst using the bath seat.
- Keep hands and fingers clear of the gaps between the slats on the seat.
- Ensure that the suckers prevent movement in the bath.

specification

Product Code	Width	Depth	Height
SBS56WH	45cm (18")	29cm (11½")	15cm (6")
SBS58WH	45cm (18")	29cm (11½")	20cm (8")
SBS512WH	45cm (18")	29cm (11½")	30cm (12")

The maximum user weight for all slatted bath seats is 190kg (30 stone)

The bath seats are constructed from sturdy and lightweight PVC plastic.