

Using your Walking Stick

Congratulations on the purchase of your Healthtech walking stick. Healthtech walking sticks are foldable and are suitable for indoor and outdoor use. They are constructed from lightweight aluminium tubing with a rubber ferrule and a choice of handles.

The ideal height for your walking stick is the distance between the ground and the crease of your wrist. To adjust the height, press the brass button below the handle. Release the button and adjust the handle until the handle locks in to the required position and the brass button can be seen again.

The sections of the walking stick are elasticated, so the walking stick should fold out automatically, as shown in the diagram below. Fold the walking stick back up when required.

Some models come with an optional elasticated strap to fasten around your wrist.

Precautions:

When folding out the walking stick, ensure you do not trap fingers or other body parts in the folded out sections. Always ensure the brass button has locked the handle in place.

Ensure that the surface you are walking on is suitable for the rubber ferrule, and that there is sufficient grip.

Periodically check the rubber ferrule was not worn out. For replacement ferrules see your Medicare Technology dealer.

Warranty

Your walking stick comes with a one year warranty from date of purchase. This covers design and manufacturing and defects only. It does not cover accidental damage or wear and tear. See your dealer for full details.

Manufacturer Details

Medicare Technology (HX5 9JP)
enquiries@medicaretechnology.com
www.medicaretechnology.com



DO NOT DISCARD THIS INFORMATION