

COLOUR-CODED PARTS:

POD LOWER ASSEMBLY:

- S08-009-01211 (RED)
- S08-009-01251 (BLUE)
- S08-009-01271 (SILVER)

FRONT SHROUD ASS'Y:

- C08-023-00111 (RED)
- C08-023-00151 (BLUE)
- C08-023-00171 (SILVER)

REAR SHROUD ASS'Y:

- C08-023-00211 (RED)
- C08-023-00251 (BLUE)
- C08-023-00271 (SILVER)

SEPERATE COMPONENTS AVAILABLE:

FROM THE TILLER POD:

- S12-009-00800 - THROTTLE POT
- M41100300000 - BATTERY GAUGE
- 3D3701100000 - KEYS
- M41610104000 - LIGHTS SWITCH
- M41610202000 - HORN BUTTON
- 3D1800500000 - IGNITION

FROM THE DRIVE ASSEMBLY:

- C09-001-10101 - 350W MOTOR
- C09-007-00300 - TRANSAXLE
- M40900600000 - EM BRAKE
- L40900600020 - CARBON BRUSH

FROM THE FRAME:

- C28-027-00200 - ANTI-TIP WHEEL
- C01-023-00100 - FRONT BUMPER

medicaretechnology
LIMITED

Ainley's Industrial Estate, Elland HX5 9JP
T: +44 (0) 1422 314488
E: enquiries@medicaretechnology.com
W: www.medicaretechnology.com/technical

TITLE:

MONAMI SCOOTER (MS030)
PARTS CATALOGUE

DATE:

16/02/07

SHEET No:

1 of 1

DRAWN:

PMK

CHECKED:

REVISION No:


-

SCALE:


NTS

DWG No:


Z40953

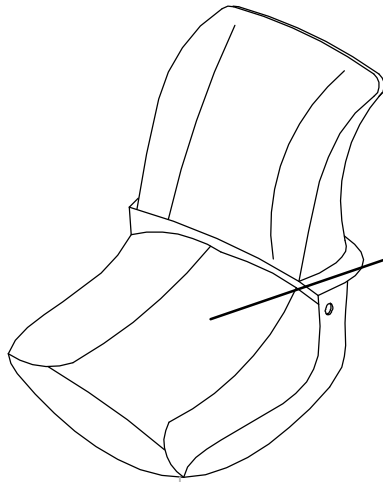

SEATM07:
SEAT ASSEMBLY

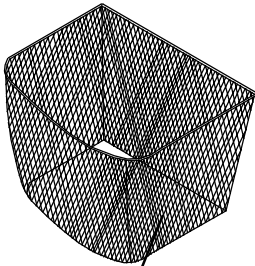

C23-009-00200:
FRONT BASKET

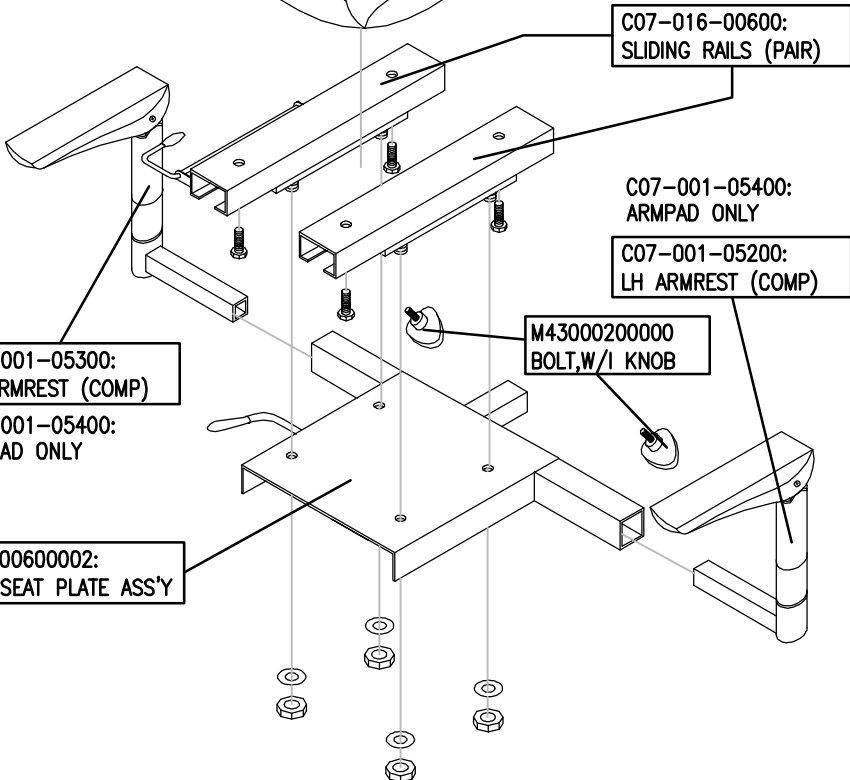

C07-016-00600:
SLIDING RAILS (PAIR)

C07-001-05400:
ARMPAD ONLY

C07-001-05200:
LH ARMREST (COMP)


M4300020000
BOLT,W/I KNOB

C07-001-05300:
RH ARMREST (COMP)

C07-001-05400:
ARMPAD ONLY

L40700600002:
STD SEAT PLATE ASS'Y

C12-001-02000
CHARGER

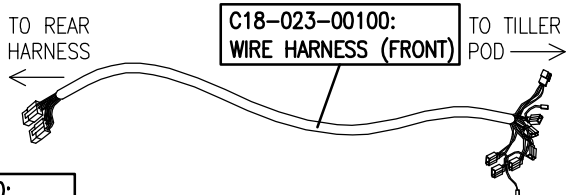

C18-023-00100:
WIRE HARNESS (FRONT)

TO REAR
HARNESS

TO TILLER
POD

C12-009-00800:
CHARGER SOCKET LEAD


C18-005-00500:
BATTERY -> CTRLR

C16-023-00100:
LEAD TO FRONT LIGHT

C16-009-00100
30A CIRCUIT BREAKER

C18-023-00200:
WIRE HARNESS (REAR)

C18-007-00900:
EM BRAKE LEAD

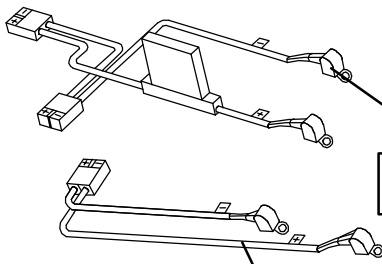

C17-029-00200:
BATTERY LEAD (+)

C17-029-00100:
BATTERY LEAD (-)

medicaretechnology
LIMITED

Ainley's Industrial Estate, Elland HX5 9JP
T: +44 (0) 1422 314488
E: enquiries@medicaretechnology.com
W: www.medicaretechnology.com/technical

TITLE:
MONAMI SCOOTER (MS030)
PARTS CATALOGUE

DATE: 16/02/07

DRAWN: PMK

CHECKED:

SCALE: NTS

SHEET No: 1 of 1

REVISION No: -

DWG No: Z40953