




Quick Start Guide Titan Power Wheelchair


Please Read Before Operating!

For technical assistance, call toll free (866) 621-4933 or (800) 544-5155.


- 1.** Remove the two knobs on the cover shroud then lift up carefully on the cover shroud to remove.

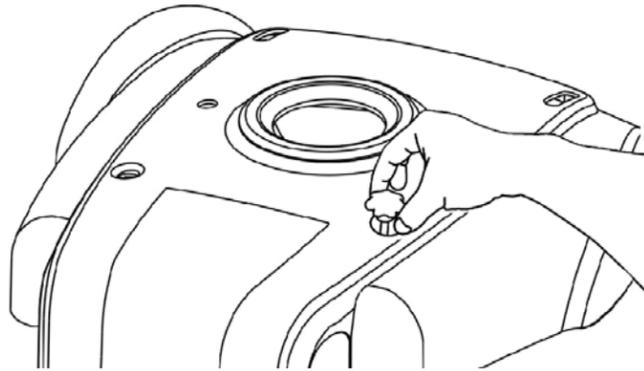


- 2.** Attach the two battery connectors to the batteries (red to red and black to black).

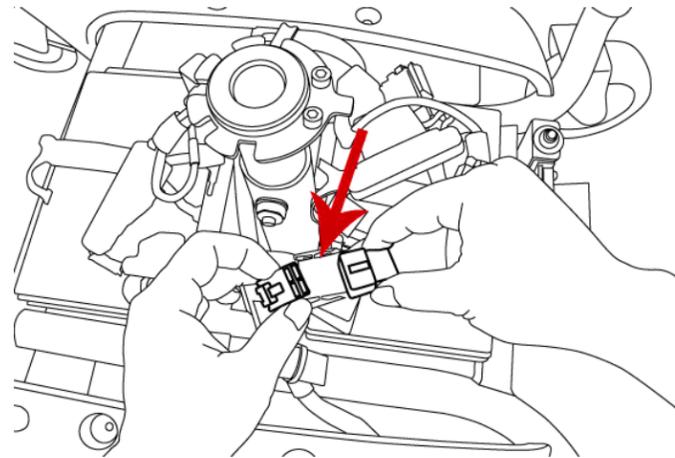


- 3.** Replace and tighten the two knobs removed in Step 1.

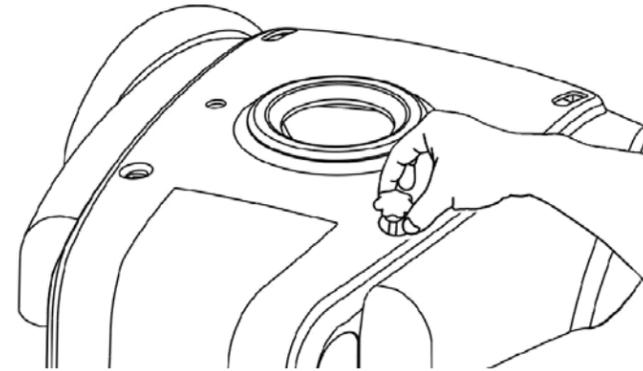


- 4.** Attach the seat by lifting up on the handle on the left side of the seat and placing the seat down on the seat post. Release the handle and rotate the seat side-to-side to ensure that the seat is locked and facing forward.

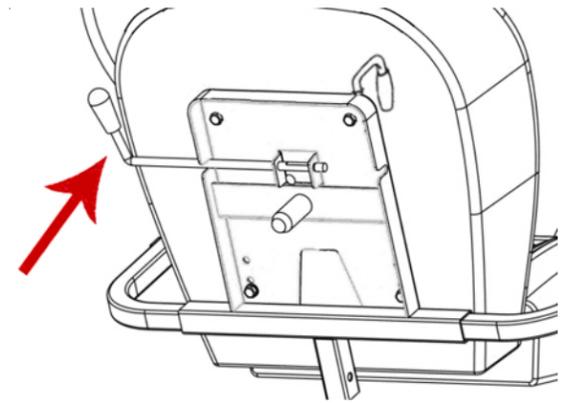


- 5.** Insert the armrests into the attachment points in the seat frame so that the armrests face forward.

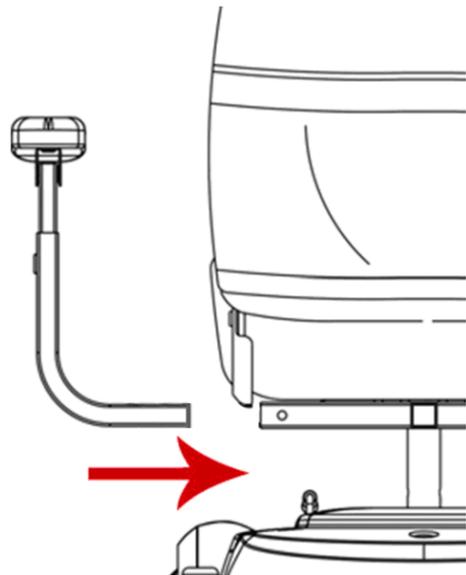


- 6.** Insert and tighten the included star knobs to lock the armrests into place at the desired width.

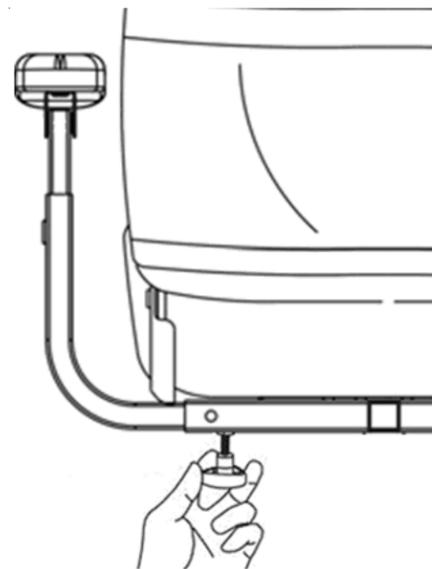


- 7.** Affix the headrest to the top of the seat by inserting the headrest posts into the holes in the top of the seat.

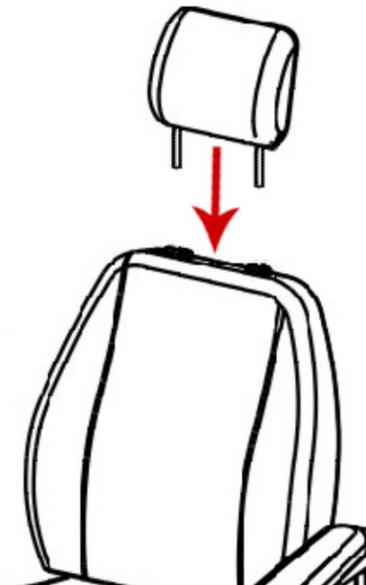


- 8.** Insert the controller's square tube into the tube under the armrest. Tighten the bolt on the underside of the controller arm tube to hold it in place at the desired extension.

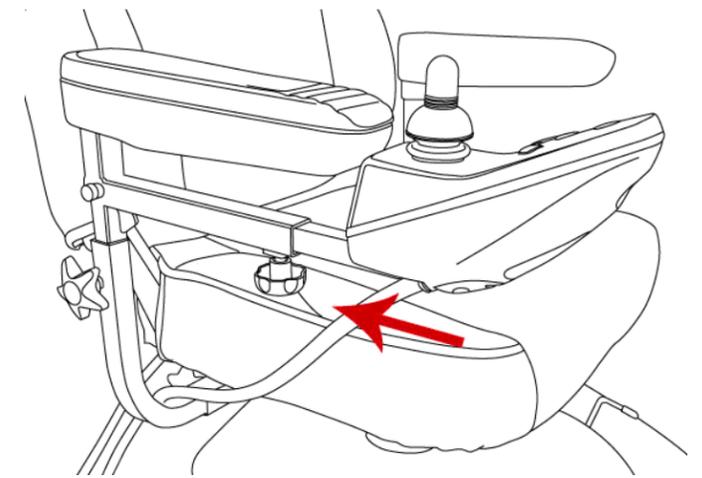

- 9.** Connect the main cable from the controller to the connection port in the product's cover shroud.

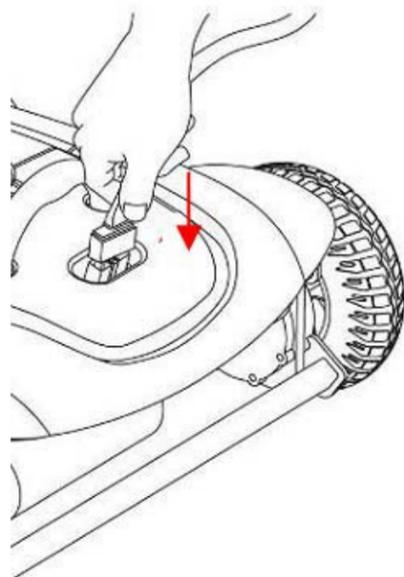

- 10.** Fully charge the wheelchair's batteries by inserting the round charger connector into the battery charging terminal on the front of the wheelchair's controller and inserting the 3-pronged plug into a standard electrical outlet.

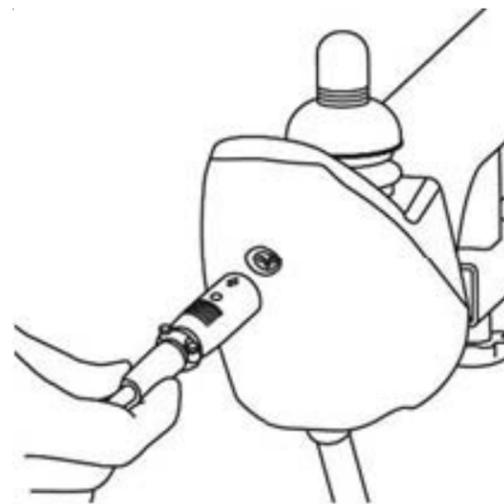

For information on disassembly for transport and on the comfort adjustments available on your power wheelchair, please consult your owner's manual.

Titan Quick Start Guide

**READ
THIS
FIRST!**

READ BEFORE UNPACKING

This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.

NOTE:The illustrations provided may not reflect your actual unit.