


The Cobalt powerchair is supplied with the two sections of the frame bolted together and the quick release locking mechanism is supplied loose in the carton.

If the a splitabe chassis is required, the quick release locking mechanism needs to be installed. The installation process is shown below:

1. Remove the seat, front body shroud and battery pack as described in the user manual.

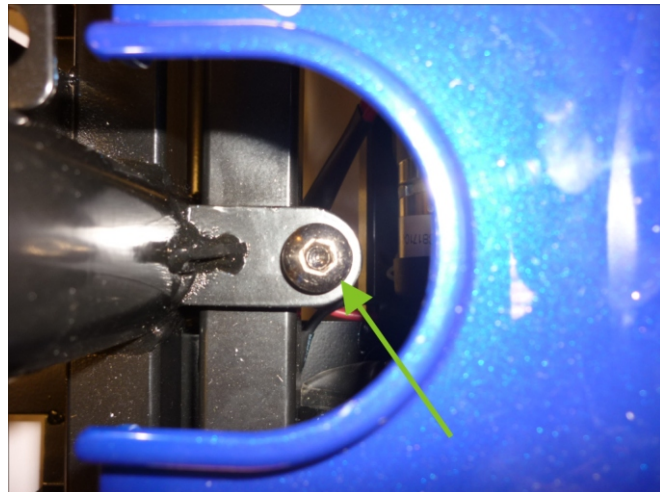

2. Remove the hex socket bolt (arrowed above) using a long handled hex key or a suitable screwdriver with bit.

Once removed, the sections of the frame can be separated.



3. Attach the quick release locking mechanism to the hole at the rear of the frame, using a 13mm spanner (see photo left).

The front and rear sections can now be assembled and split without tools as described in the user manual.