

warranty details

Your Drive branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.



Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB HX5 9JP
www.drivemedical.co.uk

Document code: DO_HANGTAG (Issued Apr 12)

**IMPORTANT: READ ALL INFORMATION
HEREIN BEFORE OPERATION**

Dolphin Bath Chair

A large, vertical version of the Drive logo, with the word "drive" in white lowercase letters on a dark blue background. The letter 'i' has a small orange square above it.



www.drivemedical.co.uk

scope


The Dolphin Bath Chair is suitable for child users who lack body control or have conditions such as lower body paralysis, hyperextension or cerebral palsy. The Chair is suitable for a single user up to 100lb (45kg).

It has a rust proof aluminium frame and easy to clean soft neoprene upholstery. It features tool free adjustment to the seat depth, and no assembly is required.

A Bath Stand to raise the Chair height in the bath is available (code DO 2010).

The Dolphin is CE marked as a class 1 medical device.

parts description

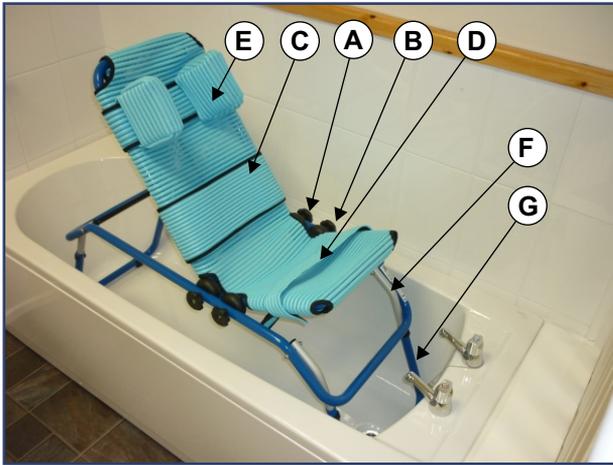

- A - Backrest Adjustment Knob
- B - Seat Adjustment Knob
- C - Torso Positioning Belts
- D - Thigh Positioning Belts
- E - Head Support Strap
- F - DO2010 extension leg
- G - DO2010 frame

bath stand

The Dolphin has an optional bath stand available to help a carer tend to the user.

Take the 'U' shaped tubing and attach it to the frame using the nuts, bolts and washers supplied. Tighten the fasteners to secure using an allen key and spanner. Repeat for the other 'U' shaped tubing.



To adjust the height of the bath stand

The height of the bath stand is adjustable. To adjust the height, remove the 'E' clips from the stand and move the tubing to the required position. Insert the 'E' to secure.

Ensure that both the front and rear of the stand are set to the equal heights.

Cleaning:

The fabric cover can be machine washed at up to 60°C. The cover and frame are not suitable for autoclave cleaning.

The cover and frame can be cleaned using a mild detergent or disinfectant. Wipe the device clean with a damp cloth and dry with a towel.

Checks:

The device should be periodically checked for loose fasteners, damaged or torn fabric, frame and paint damage. If any of these problems are found the device should not be used until the device or part is repaired or replaced.

specification

| | |
|----------------------|-----------------------------|
| Length: | 100 - 113cm (39.5" - 44.5") |
| Seat Depth: | 33 - 46cm (13" - 18") |
| Weight: | 2.5kg (6lb) |
| Height from floor: | 15cm (6") |
| Back Height: | 61cm (24") |
| Width: | 43cm (17") |
| Weight limit: | 45kg (100lb) |
| Stand adds Height: | 28 - 46cm (11" - 18") |
| Weight (with Stand): | 4.5kg (10lb) |

safety precautions

- The Dolphin should only be operated by a carer. Under no circumstances should a user be left attended with the device.
- The Dolphin is suitable for indoor use only and should only be used on a flat and level surface.
- Do not allow a user to occupy the device if they are too small or too large for the device.
- Do not exceed the weight limit.
- Ensure the user is securely positioned in the device.
- Always ensure that all straps are correctly adjusted. Incorrectly adjusted straps can cause risk of suffocation.
- Note: All recommendations contained herein should be checked with your health professional (such as occupational or physio therapist) before use.

adjustments

Backrest Angle Adjustment. Loosen knob (A) as shown below. Repeat for the other side. Tilt the backrest to the required position. Retighten both knobs to secure the backrest in position.



Legrest Angle Adjustment. Loosen knob (B) by turning. Repeat for the other side. Tilt the legrest to the required position. Retighten both knobs to secure the backrest in position.

Legrest Depth Adjustment. Locate the two push buttons on the legrest (this may require you to release the velcro on the legrest section of the fabric cover and pull the cover down).

Press both brass push buttons (as shown below) and pull the legrest in or out until the required depth is reached. Release the buttons and keep pulling the section until it locks in position and the two brass push buttons can be seen.



Support Straps. The support straps (C, D and E) can be placed anywhere on the bath chair. Securely engage the hook and loop of the strap once positioned. The head cushions on the Headrest Support Strap can slide along the strap for width adjustment.

care and maintenance



Fabric Cover Removal.

Alter the backrest and legrest sections so they are upright (as shown).

Then release the hook and loop flaps on the head and leg securing the fabric (as shown below). Slide the cover up the frame to remove.



Fabric Cover Installation.

The installation procedure is the removal procedure in reverse. Firstly, adjust the backrest and leg section to they are upright (as shown previously) and slide the cover over the frame.

Secure the cover by installing the hook and loop straps. Then adjust the backrest and leg rest to the required angle.