



**Back Up Battery Installation Guide – Riser Chairs (Lift Chairs)**

Many of the Drive Rise and Recline armchairs come with a battery operated reset facility. This will allow the user to install disposable batteries so that the chair will lift or return to its normal position in the event of a power cut. Batteries should be replaced annually or when the reset facility has been used once.

The battery backup functions on each chair are described below:

Code	Desc	Battery Backup
925SABE	Sasha	Yes – In Separate Power Supply
928WSMBE	Star	Yes – In Separate Power Supply
922BABE	Sofia	Yes – In Separate Power Supply
CLR11	Chicago	None
CLR19	Indiana	Yes – In Separate Power Supply – from 06/15
920BABE	Serena Button Back	Yes – In Separate Power Supply
920WABE	Serena Waterfall Back	Yes – In Separate Power Supply
920SABE	Serena Standard Back	Yes – In Separate Power Supply
924SABE	Sienna	Yes – In Separate Power Supply
927SSMBE	Santana	Yes – In Separate Power Supply
CLR3Y	Boston	Yes – In Separate Power Supply
CLR12	Nevada	Yes - In Control Box
CLR12Y	Portland	Yes - In Control Box
CLR24/34	Memphis	Yes - In Control Box
ILR4	Seattle	Yes – In Separate Power Supply
ILR12	Dakota	Yes – In Separate Power Supply
923WABE	Serena	Yes – In Separate Power Supply
ALC2205	Lars Single Motor	Yes - In Control Box
ALCLD01	Lars Dual Motor	Yes - In Control Box
927SDMBE	Santana Dual Motor	Yes – In Separate Power Supply

## **Back Up Battery Installation – Chairs with Separate Power Supplies**

*Applies to chairs:*

*Boston from 2013 (CLR1Y and CLR3Y), Borg (BRC001 and BRC002), Oakland (CLR23), Dakota (ILR12Y), Intalift (ILR4), Indiana from 06/15 (CLR19 and CLR29).*

*Sasha (925SABE), Star (928WSMBE), Sofia (922BABE), Serena (920BABE, 920WABE, 920SABE), Sienna (924SABE), Santana (927SSMBE and 927SDMBE).*

*Batteries Required: 2x 9V PP3.*



1. Remove the battery compartment lid from the power supply by pushing the lid away from the power supply.



2. Pull out the two battery connectors from the compartment in the power supply and push a 9V battery on to each of the connectors.



3. Place the batteries and wiring inside the battery compartment of the power supply.



4. Slide the battery compartment lid back on to the power supply.

## **Back Up Battery Installation – Chairs with Control Boxes**

*Applies to: Nevada and Portland (CLR12 and CLR22), Memphis (CLR12), Denver (CDL8), Boston (prior to 2013), Chicago (Prior to 2011), Lars (ALC2205)*

*Requires 2x 9V PP3 batteries:*




1. Raise the chair and lift up the fabric access flap at the rear to locate the control box. Using your finger nail, lift the locking tab on the bottom of the battery compartment lid and release.



2. Reach in to the battery compartment and pull out the two battery tab terminals from the compartment.



3. Connect two 9V batteries to the tab terminals pulled out from the battery compartment.

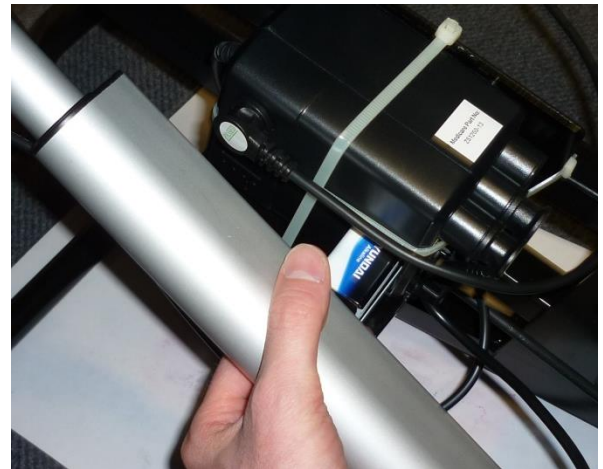

4. Push the two batteries back in to the compartment and reattach the battery compartment lid.

© Drive Medical Ltd  
Ainley's Industrial Estate,  
Elland, West Yorkshire, UK HX5 9JP.

Issued 06/15