



PRODUCT INSTRUCTION SHEET



REFERENCE Bosworth Shower Bench (127) and Bosworth Shower Bench @ 18” (127-18)

DESCRIPTION

Adjustable height bench for use in the shower.
Heavy duty perforated injection moulded plastic seat.
Seat clips onto frame and can be removed
for ease of cleaning.



TECHNICAL SPECIFICATION

Width between arms: 19" (49cm)
Seat size: 15" x11" (380mm x 280mm)
Width: 21"
Depth: 16"
Height adjustment range: 20" – 26"

USAGE REQUIREMENTS

1. Do not exceed recommended weight 160kg (25 Stones). The bench is not designed to be dropped upon.
2. When adjusting legs ensure that there is no gap between the horseshoe clip and frame. The clip spindle must be fully located right through the outside frame.
3. Periodically, check legs for total movement.
4. Ensure seat is fully located on each cross member.
5. Clean frame with cream cleaner. Clean seat with soap and water.