

## Warranty Details

This device was built to exacting standards and carefully inspected prior to shipment. The two year warranty is an expression of our confidence in the device.

Your Drive product is warranted to be free of defects in materials, workmanship and design for two years from the date of purchase.

This warranty does not include defects as a result of (amongst others) accidental damage, wear and tear or failure to comply with these instructions.

In the event of a defect within the warranty period, contact the stockist from whom you purchased the device to apply the warranty.

This warranty is offered free-of-charge to consumers by Drive Medical Ltd. For full warranty conditions please visit the Drive Medical website [www.drivemedical.co.uk](http://www.drivemedical.co.uk)

This warranty does not affect your statutory rights.



Drive Medical Ltd,  
Ainley's Industrial Estate,  
Elland, GB HX5 9JP  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

Doc Code: 1206\_IFU (Iss 05/14)

# Raised Toilet Seats



## Installation Instructions

1. Remove the product from the carton and read instructions carefully before installing the raised toilet seat.
2. Remove the toilet seat and lid from the toilet (*optional*)
3. Turn both locking clamps counter clockwise until they can no longer turn. This will open the locking devices.
4. Place the raised toilet seat on the toilet bowl and position in a secure fashion. Be sure that the two locking brackets are on both sides of the bowl wall and the seat is flush to the bowl.
5. Turn the locking brackets clockwise until tight. Hand tighten only. Do not use tools as threads on the adjustment brackets may strip.
6. Check for secure fit routinely before using the product. The user should be centred over the toilet seat.

## Removal Instructions

1. Rotate the locking brackets counter clockwise until the seat is loose.
2. Lift the front of the raised toilet seat and pull forward and upward in one move.

## Spare Parts Available

The locking brackets for the raised toilet seat are available to purchase as a spare part from your Drive Medical dealer. The product code for this part is Z1206BKT

## Cleaning Instructions

The raised toilet seat can be cleaned with a non-abrasive detergent or household cleaner with warm water.

The raised toilet seat can also be cleaned with a diluted disinfectant.

When washed, rinse the raised toilet seat with warm water and wipe with a dry cloth.

The seat can be thermally decontaminated in at 80°C for up to 15 cycles.

**Avoid using abrasive cleaners and/or cloths as this can cause damage to the raised toilet seat.**

## Safety Precautions

Always check that the seat is correctly and securely locked in place before using.

Ensure that the locking brackets are tightly touching the outside edges of the toilet bowl.

The raised toilet seat may not be suitable for use with all shapes of toilets.

If the raised toilet seat feels unstable or can be moved by hand, do not use.

The user should always keep their weight centrally over the toilet. Leaning excessively to either side, forwards or backwards could cause instability and risk of injury and damage.

Do not exceed the maximum user weight of 200kg (32 stone)