


drive

DeVilbiss
HEALTHCARE

Red Day Clock

Instructions for Use

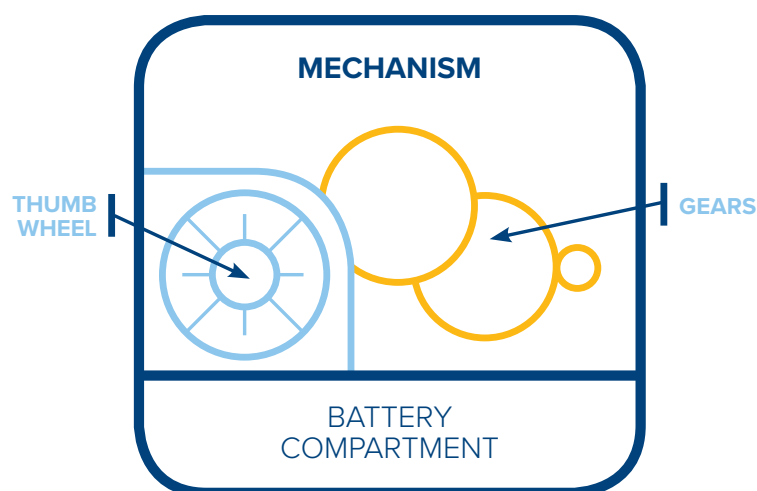

Installing the Battery

(1 x 'AA') not included

1. Battery must be inserted in the CORRECT direction. Follow the diagram in the mechanism's compartment. The positive end (+) of the battery should be inserted first.
2. You will NOT hear the expected "tick" "tick" "tick" sound.
3. Look at the gears through the mechanism's transparent case. Exactly once EVERY 14 SECONDS you will see the gears move. If they do not move it means the battery is dead or it was put in backwards.



Setting the Pointer

1. Use the thumb wheel on the mechanism to move the black pointer hand where you want it.
2. To get started set the pointer anywhere within today's day. (Our photo assumes that it is noon on a Saturday).
3. Notice that each day is divided into two halves.

Midnight to Noon Noon to Midnight

Noon is marked by a line in the centre of each day. In the middle of the first half is 6am and the middle of the second half is 6pm (These points are not actually marked on the dial).

4. You can therefore place the pointer to the APPROXIMATE position corresponding to the current time. You can wait until the current time is exactly noon, then set the pointer accurately.

CE

Drive DeVilbiss Healthcare Ltd, Heathfield Lane,
Birkenshaw, West Yorkshire, BD11 2HW

www.drivedevilbiss.co.uk