

# 4-WHEEL FOLDING SCOOTER

## QUICK START GUIDE

Follow the instructions below to quickly and safely set up and start your scooter.



1. Unpack the scooter, as shown above. You will have the following parts:

**1 x scooter unit**

**1 x battery pack**

**2 x keys**

**1 x remote fob**

**2 x armrest**

**2 x threaded knob for armrest**

**1 x charger and mains cable**


2. Plug the mains lead in to the charger, and then plug the round connector from the charger in to the charging socket of the scooter.  
Finally switch the charger on at the mains.



3. Insert the key in to the ignition (as shown below) and switch the scooter on. Then press the right hand side button (arrowed) for 1 second. The scooter will then slowly beep and unfold. For safety, the unfold process will stop halfway. Press the button again to fully unfold the scooter. When fully unfolded the scooter will emit three quick beeps.



4. Insert the battery pack as shown below.

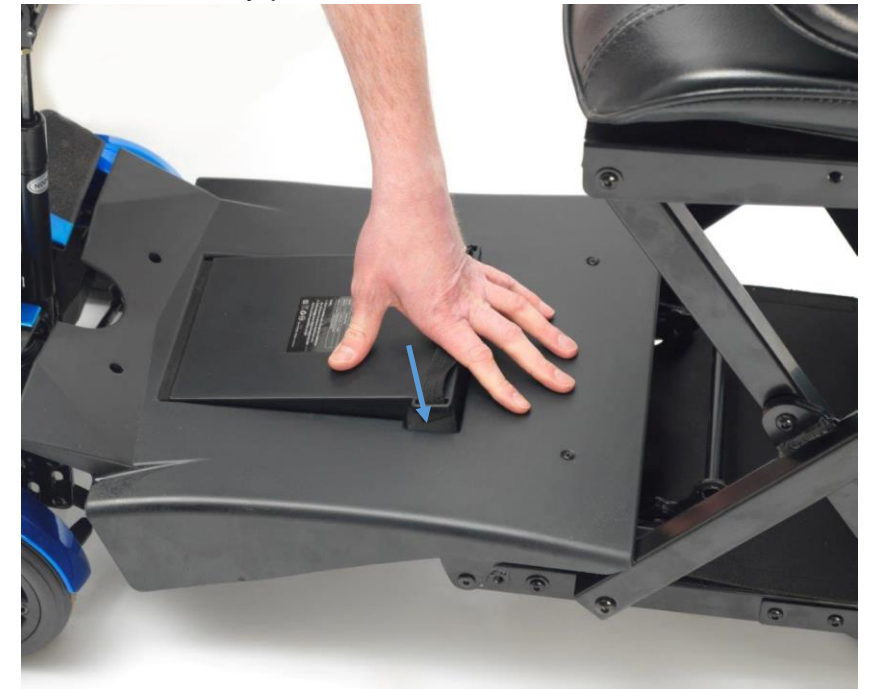

5. Flip the backrest in to position.



**PLEASE TURN OVER**

6. Pull on the catch (arrowed) on the tiller to unlock. Then pull up the tiller until it is set at the correct height for the user. Push the catch to lock the tiller.



7. Adjust the tiller angle by pressing the lever (arrowed) whilst pulling or pushing the tiller to the correct angle. Release the lever to lock the tiller in position.



8. Insert the armrest in to the receiver.



9. Secure the armrest by screwing in to the threaded knob in to the hole (arrowed).



10. Insert the key in to the ignition and turn to start the scooter.



#### 11. CHARGE BEFORE USE

**Plug the mains lead in to the charger, and then plug the round connector from the charger in to the charging socket of the scooter. Finally switch the charger on at the mains. The scooter can be charged either unfolded or folded (refer to user manual for folding instructions).**

