

# Auto-Fold Scooter Quick Start Guide

Issued 2019/05

**ALWAYS FULLY READ THE  
OWNER'S HANDBOOK  
SUPPLIED WITH THE SCOOTER  
BEFORE DRIVING**

1. Remove the insulating layer of foam between the battery pack and scooter (arrowed).



2. Turn the tiller adjustment knob (shown below) anti clockwise to loosen, and extend the tiller approximately 1" (2cm).



3. Push the battery pack backwards so it connects to the scooter.



4. The folding button should illuminate RED. Press the button for one second and release to fold out the scooter.



5. Install the carpet in the footwell. The carpet attaches with Velcro.
6. Plug the charger in to the battery pack or the scooter and begin charging.

*Note: Fully charge the battery before use. The rest of the installation can be done whilst the scooter is charging.*

7. Turn the tiller adjustment knob anticlockwise to loosen the tiller handlebar (if required). Then pull up on the tiller until it reaches the desired height. Turn the adjustment knob clockwise to secure the tiller.



8. Flip up the backrest.



9. Press the button above the reflector whilst flipping up the armrest. Ensure the armrest locks in position.



10. Adjust the height of the armrests to suit the user, by pressing the button underneath the armpad whilst lifting up the armpad.



11. When charging is complete remove the charger. Then press the ON button on the tiller dash when the keyless fob is close.

