

PRODUCT INSTRUCTION SHEET

REFERENCE

Belmont PU Corner Shower Stool (124)

DESCRIPTION

- Adj. Ht. triangular shower stool which fits neatly into the corner of the shower base allowing full access for the carer.
- Tough, plastic coated (peel resistant), tubular steel frame.
- Comfortable PU (polyurethane) moulded padded seat.

TECHNICAL SPECIFICATIONS

Seat Height Min:	51cm (20")
Seat Height Max:	64cm (25")
Seat Width:	40cm (16")
Seat Depth:	36cm (14")
Footprint:	42.5cm (16 ¾") x 42.5cm (16 ¾")
Product Weight:	3.5Kg (8lbs)
Max User Weight:	165Kg (26st)



FITTING REQUIREMENTS

- Place in shower.
- Ensure all three legs are level and in contact with the shower floor.
- Sit centrally on the shower stool.

HEIGHT ADJUSTMENT

- Put seat upside down on floor/flat surface.
- Adjust to the correct height by removing pin clips, sliding extension leg to the required position and replace pin clip.
- Ensure all three leg extensions are secured to the same adjustment hole to ensure even height.

SAFETY PRECAUTIONS

- Do not exceed the maximum user weight limit.
- Inspect at regular intervals for wear and damage.
- Do not use if rubber feet are damaged.

MAINTENANCE

- Clean regularly, by wiping down with a mild detergent /disinfectant solution.
- This product is intended for use in a wet room / shower. It is not intended for use outside and should not be immersed in water for long periods of time. Ensure the product is dried thoroughly after use.