

## **PRODUCT INSTRUCTION SHEET**

**REFERENCE** Bosworth Shower Bench (127) and Bosworth Shower Bench @ 18" (127-18)

### **DESCRIPTION**

Adjustable height bench for use in the shower.  
Heavy duty perforated injection moulded plastic seat.  
Seat clips onto frame and can be removed  
for ease of cleaning.



### **TECHNICAL SPECIFICATION**

Width between arms: 19" (49cm)  
Seat size: 15" x 11" (380mm x 280mm)  
Width: 21" (53cm)  
Depth: 16" (41cm)  
Height adjustment range: 20" – 26" (51 – 66cm)

### **USAGE REQUIREMENTS**

1. Do not exceed recommended weight 160kg (25 Stones). The bench is not designed to be dropped upon.
2. When adjusting legs ensure that there is no gap between the horseshoe clip and frame. The clip spindle must be fully located right through the outside frame.
3. Periodically, check legs for total movement.
4. Ensure seat is fully located on each cross member.
5. Clean frame with cream cleaner. Clean seat with soap and water.
6. This product is intended for use in a wet room / shower. It is not intended for use outside and should not be immersed in water for long periods of time. Ensure the product is dried thoroughly after use.