

## **PRODUCT INSTRUCTION SHEET**


### **REFERENCE**

Allexton Rigid Blanket Cradle (138)

### **DESCRIPTION**

Plastic coated continuous looped blanket support, placed between mattress and bed base.

The weight of the bed clothes is supported, keeping them clean off the patient for ventilation and pressure relief.

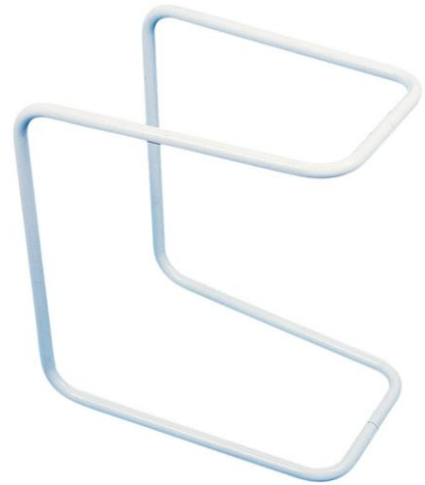

### **TECHNICAL SPECIFICATIONS**

Overall Width:	11" (28cm)
Overall Height:	20" (50cm)
Length of base support:	17" (44cm)
Product weight:	1.8 kg (4lb)

### **USAGE REQUIREMENTS**

NB: No maximum user weight/load is given for this product as the limit is dependent upon the actual weight of the bed clothes. We recommend a risk assessment is carried out before use.

1. Ensure the mattress is flat, and the cradle is horizontal or parallel to the top of the mattress.
2. When positioning the unit, make sure it is pushed fully under the mattress and the vertical struts rest against the mattress edge.
3. Check the position of the product regularly to maintain its correct position in line with the original risk assessment
4. Ensure that the coating is not damaged for hygiene purposes.

### **CLEANING**

The product is designed for minimum maintenance and can be cleaned with a soft cloth using general neutral detergent and warm water. Do not use harsh abrasive materials. To disinfect, wipe over with a mild chloride disinfectant (solution of 1000 parts per million) Autoclavable at 80C