

## PRODUCT INSTRUCTION SHEET

### PRODUCT CODE

Foxton Shower Chair Adj Ht (140A)  
Foxton Shower Chair Adj Ht Det Arms (140AD)

### DESCRIPTION

Nylon coated all steel welded frame with `H` section base for added strength. Moulded polypropylene seat with drainage holes. Adjustable in height. Available with fixed or removable arms.



### TECHNICAL SPECIFICATIONS

Internal width between arms	19"
External width:	21"
Seat Width x Depth:	17" x 15"
Floor to seat height (140A-140AD)	18" - 23"
Overall product height (140):	32.5"
Product Weight:	5.4kg (140A) 7.2Kg (140AD)
Weight Capacity:	165kg (26st)



### USAGE REQUIREMENTS

1. Where height adjustment is required, ensure that there is no gap between horseshoe clip and frame. The clip spindle must be fully located right through to the outside of the frame.
2. OTs should do a risk assessment on any appliance at time of issue, ensuring the products surface stability.
3. Do not exceed weight limit of 165kg (26 Stone) and ensure that patients do not 'swing-on' the frames either in a forward or backward motion.
4. Where detachable arms are fitted ensure they are fully located in the retaining tubes on either side of the frame.
5. Clean frame with cream cleaner, seat with soap and water.
6. This product is intended for use in a wet room / shower. It is not intended for use outside and should not be immersed in water for long periods of time. Ensure the product is dried thoroughly after use.



## **PRODUCT INSTRUCTION SHEET**

### **WARRANTY**

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement. If you have any queries or require full warranty conditions, please contact your Drive stockist. This does not affect your statutory rights.