

**PRODUCT INSTRUCTION SHEET**

**REFERENCE:** Foxtan Fixed Height Stackable Shower Chair (140S)

**DESCRIPTION**

Nylon coated all steel welded frame with a moulded polypropylene seat with small drainage holes. This product has fixed arms and is fixed in height. This product can be stacked for easy storage.

**TECHNICAL SPECIFICATION**

Seat width at widest part:	18"
Seat Depth:	14"
Floor to Seat Height at front:	15 ¼"
Footprint (WxD):	23" x 18.5"
Internal width between arms:	21.5"
External width:	23 ¼"
Overall Depth:	19"
Maximum user weight:	26st (165Kg)



**USAGE REQUIREMENTS**

1. Do not exceed weight limit of 127kg (20 Stone) and ensure that patients do not 'swing-on' the frame either in a forward or backward motion. The 4 ferrules must be in contact with the floor at all times.
2. Clean frame with cream cleaner and the seat with soap and water.
3. This product is intended for use in a wet room / shower. It is not intended for use outside and should not be immersed in water for long periods of time. Ensure the product is dried thoroughly after use.



## **PRODUCT INSTRUCTION SHEET**

### **WARRANTY**

Your Drive DeVilbiss branded product is warranted to be free of defects in materials and workmanship for one year from date of purchase. This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer. In the event of a defect covered by this warranty, we will at our option supply parts or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement. If you have any queries or require full warranty conditions, please contact your Drive stockist. This does not affect your statutory rights.