

PRODUCT INSTRUCTION SHEET

REFERENCE

Kibworth Toilet Frame Heavy Duty (156HD)
With Floor Fixing Kit (156HDFK)

DESCRIPTION

Bariatric free standing rigid double strut steel frame with Duradip peel resistant plastic coating and polyurethane (PU) Armrests.

TECHNICAL SPECIFICATIONS

| | |
|--------------------------|-------------------------|
| Frame width: | 72cm (28") |
| Depth: | 52cm (20") |
| Width between arms | 63cm (25") |
| Height adjustable range: | 69cm (27") – 82cm (32") |
| Product Weight: | 7kg (15lb) |



USAGE REQUIREMENTS

1. Both free standing and floor fixed models are positioned around the toilet bowl.
2. When adjusting the legs, ensure that there are no gaps between horseshoe clip and frame. The clip spindle must be fully located right through the outside frame.
3. Periodically check for elongated or stressed adjustment holes. Check for split, worn or loose ferrules. Check all fastenings are secure
4. Ensure the legs are adjusted at equal height.
5. If floor fixings are used, for greater stability, ensure that 3 fixing screws are placed in each flange. The screws can be either standard wood screws and/or rawl plug fittings.
6. Frames require an occasional wipe over with a soft cloth- use general purpose neutral detergent and warm water. Do not use harsh abrasive materials. To disinfect, clean as above and wipe over with a solution of 1000 parts per million chlorine disinfectant (haztabs/precept), rinse and dry.

MAXIMUM USER WEIGHT: 254Kg (40st)