



PRODUCT INSTRUCTION SHEET

REFERENCE

Kibworth Toilet Frame with PU Arms (156PU)
 Kibworth Toilet Frame with Floor Fixing Kit and PU Arms (156PUF)

DESCRIPTION

All steel welded frame, nylon coated with rigid double-strut design. Moulded plastic armrest. Adjustable in height. Maximum user weight 180kg (28st).

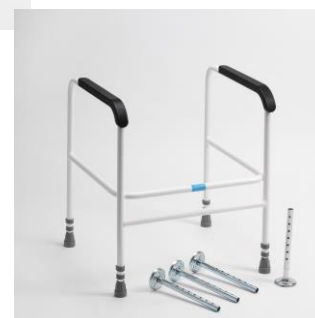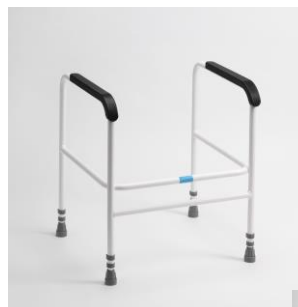

TECHNICAL SPECIFICATIONS

| | |
|--------------------------|--------------|
| Outside Frame width: | 51cm (20") |
| Width between Arms | 43cm (17") |
| Height adjustable range: | 25"-31" |
| Product Weight: | 4kg (9lb) |
| Weight Capacity | 180kg (28st) |

USAGE REQUIREMENTS

1. Both free standing and floor fixed models are positioned around the toilet bowl.
2. When adjusting the legs, ensure that there are no gaps between horseshoe clip and frame. The clip spindle must be fully located right through the outside frame.
3. Periodically check for elongated or stressed adjustment holes. Check for split, worn or loose ferrules. Check all fastenings are secure.
4. Ensure the legs are adjusted at equal height.
5. If floor fixings are used, for greater stability, ensure that 3 fixing screws are placed in each flange. The screws can be either standard wood screws and/or rawl plug fittings.
6. Frames require an occasional wipe over with a soft cloth- use general purpose neutral detergent and warm water. Do not use harsh abrasive materials. To disinfect, clean as above and wipe over with a solution of 1000 parts per million chlorine disinfectant (haztabs/precept), rinse and dry.
7. Clean frame with cream cleaner.