

IMMY™ Cryptococcal Antigen (CrAg) Lateral Flow Assay (LFA) Test




Add **1 drop** of specimen diluent to tube




Add **40 µL** of blood/CSF



Insert blue end of the strip into tube

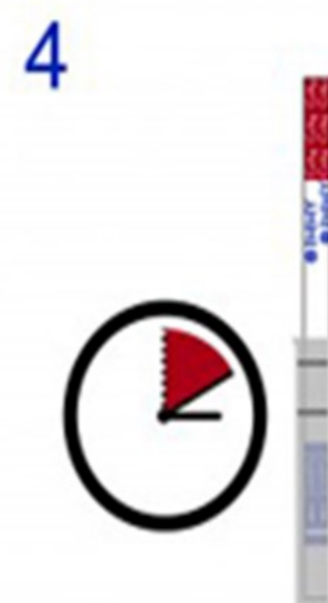

Incubate for **10 minutes**

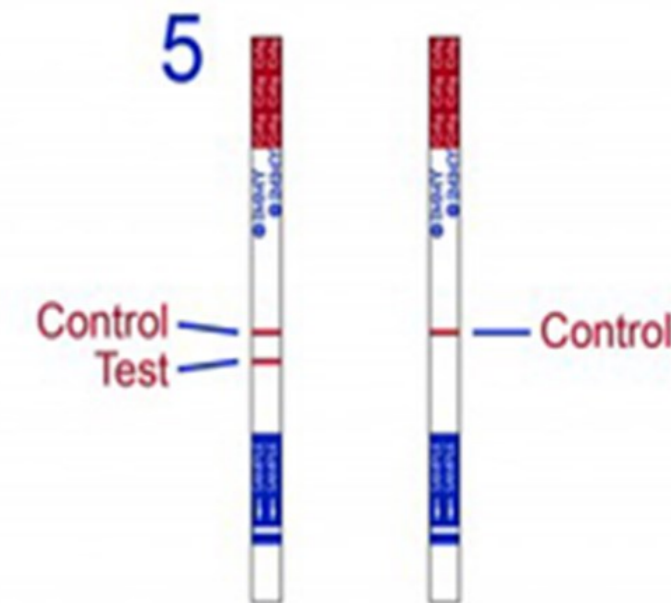

1 line = **Negative**

2 lines = **Positive**

Points to remember:

- Use a new pipette tip and fresh gloves for each test.
- Ensure the pipette volume is set correctly.
- Do not read a result beyond 10 minutes.
- Ensure control line present or else the test is invalid.