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Congratulated with the purchase of the Looky.

Using for the first time

It is necessary to charge the batteries before using the Looky. Place the batteries supplied into the battery compartment (6) with the correct orientation. Connect the adapter supplied into the adapter connection (4) and plug into the mains socket. Charging takes approximately 4 hours.

Switching on the Looky

Press either the yellow button (1) or the blue button (2) for about 3 seconds until a two tone beep is audible to switch on the Looky.

During operation a short beep (3) is audible when either button is pressed.

Freeze function (1)

Press the yellow button (1) to activate the freeze picture function. To unfreeze, press the yellow button (1) again.

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It is possible to change the picture mode using the blue button (2) while the freeze picture function is activated.

Picture Mode (2)

There are 3 picture modes available. These can be selected by pressing the blue(2) button. The modes are:

- Photo mode (colour)
- Positive mode (black-white)
- Negative mode (white-black)

Writing mode

It is recommended that Photo mode is selected when using the Looky as a writing aid.

Switching off the Looky

Press either the yellow button (1) or the blue button (2) for about 3 seconds until a two tone beep is audible to switch off the Looky.

If the Looky detects no movement within 3 minutes then it automatically powers off.

A two tone beep is audible when this happens.

Charging the Batteries of the Looky

When the batteries are empty the Looky emits a warning beep. A high tone is audible and this then fades away.

To charge the batteries connect the adapter supplied into the adapter connection (4).

Two short beeps are audible.

(Attention! Do not use any other adapter other than the one supplied and specified for use with the Looky)

A green LED (5) burning continuously indicates the Looky is charging.

A green LED (5) flashing indicates the batteries are charged.

Charging of the Looky takes approximately 4 hours. After charging the batteries, you can use the Looky for up to 1,5 hours continuously.

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(Attention! You can use the Looky while it is charging, but it will take longer to complete the charging process).

(Attention! It is not recommended to charge the batteries in an external battery charger. There is no guarantee that the batteries will be fully charged).

It is possible to use Alkaline batteries (type AA) in the Looky but the duration of use is significantly shorter.

If the adapter is connected to the Looky containing Alkaline batteries in error, a continuous two tone beep is emitted and the LED (5) flashes on and off. This indicates that the batteries will not be charged. Remove the adapter first and then the alkaline batteries and replace with rechargeable ones.

(Attention! Only replace with same type as originally supplied)

What to do if problems occur

Disconnect the adapter if attached.

Remove the batteries from the battery compartment (6).

Wait for 3 minutes and replace the batteries.

Switch back on the Looky.

If problems persist, please contact your nearest dealer.

Safety Precautions

Protect the device and the batteries from excessive moisture, rain, sand or extreme temperatures.

Do not cover the device when charging the batteries.

Clean the device with a clean, dry cloth. Do not use cleaning products that contain alcohol, methylated spirits, ammonia or that are abrasive as these can damage the device.

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Copyright 2006 by Rehan Electronics.

All rights reserved.

No part of this publication may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying and recording, or by any information storage or retrieval system without the prior written permission of Rehan Electronics.

Notice

Rehan Electronics makes no guarantees of any kind with regard to this manual. Rehan Electronics shall not be held liable for errors contained herein or for consequential or incidental damages incurred as a result of acting on information contained in the manual.

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Looky logo is a trademark of Rehan Electronics