

# User Manual

## MagniLink Vision TTS2

LVI Low Vision International

Verkstadsgatan 5

SE-352 46 Växjö

Tel: +46 470-727700

Faks: +46 470-727725

[info@lvi.se](mailto:info@lvi.se)

[www.lvi.se](http://www.lvi.se)



## Table of Contents

1	About LVI.....	5
2	Introducing MagniLink Vision TTS.....	6
3	Information.....	7
3.1	Reuse, installation and maintenance.....	7
3.2	Cleaning .....	7
3.3	Safety information.....	7
4	Unpacking .....	9
4.1	Packaging.....	9
5	Installation .....	10
6	Use .....	11
6.1	Panel .....	11
6.2	Reading table.....	13
6.3	Profiles and settings .....	14
6.4	Simple profile.....	15
6.5	Advanced profile .....	17
6.6	Settings.....	22
6.7	Quick guide computer mouse.....	28
6.8	Computer mouse.....	29
6.9	Extra voices on a USB stick .....	29
6.10	Quality of OCR conversion .....	30
7	Ergonomics .....	31
8	Troubleshooting .....	32
9	Technical information.....	33
10	Declaration of Conformity .....	35



## 1 About LVI

LVI Low Vision International AB was founded in 1978 and is one of the world's leading manufacturers of equipment for persons with visual impairments. Development of new products takes place in close cooperation with users and professional staff within the low vision rehabilitation industry. LVI is quality-certified in accordance with ISO 9001 and environmentally-certified in accordance with ISO 14001.

The information in this manual has been carefully checked and is believed to be accurate. However, LVI assumes no responsibility for any inaccuracies that may be contained in this manual. In no event will LVI be liable for direct, indirect, special, exemplary, incidental, or consequential damages resulting from any defect or omission in this manual. In the interest of continued product development, LVI reserves the right to make improvements in this manual and the products it describes at any time, without notice or obligation.

## 2 Introducing MagniLink Vision TTS

Thank you for choosing this video magnifier. Our aim is for you to be satisfied with your choice of supplier and reading aid.

MagniLink Vision TTS is a combined reading camera and reading machine. The reading camera includes features such as autofocus, several artificial colours and the potential to set the magnification. The reading machine makes it possible to OCR-process a text and have it read aloud.



MagniLink Vision meets all the requirements for CE marking of medical devices in accordance with Directive 93/94/EEC. Our products comply with MDD 93/94/EEC with additions from Directive 2007/47/EC.

## 3 Information

### 3.1 Reuse, installation and maintenance

The device may only be technically overhauled and prepared by an authorized company. Re-use may only be carried out after appropriate inspection and cleaning / disinfection.

### 3.2 Cleaning

Before equipment cleaning and maintenance note:

- Disconnect the device from the power supply before cleaning.
- Use a slightly damp (not wet) cloth and a cleaning / disinfectant agent intended for monitors and computers.
- Do not allow any liquids to enter the device.

### 3.3 Safety information

- Only use the power adaptor supplied with the product. Using an incorrect type of power adapter may cause damage to the system.
- Never operate the product if it has been exposed to water or other liquids. Damaged or wet product can cause electric shocks or fire.
- Place the power cord in such a way that you can neither trip over it or that crush damages can occur.
- Place the system at least 10 cm from the edge of the table. When moving the reading table, make sure your hands are not below or behind the reading table, as there is a risk of your fingers becoming trapped. When the reading table is in its outermost position, an excessive load can cause the system to tip over, resulting in personal injury.
- If the system has to be moved, do not lift at the screen or reading table as these are moving parts.
- If the product is moved from the cold into a warm room, condensation (moisture) can form inside the unit or on the camera lens. To prevent problems with condensation, the apparatus should be installed in the room at least 30 minutes prior to usage.
- If the system requires cleaning, detach all cables. Use a slightly damp (not wet) cloth to clean the system. Only use water or cleaning / disinfectant agent intended for monitors and computers.

- Do not rely 100 per cent on the results of an OCR conversion; the OCR engine can misinterpret the text. This is particularly important to bear in mind when processing sensitive information such as a medical prescription.

If in doubt, contact your supplier.

#### Warning



The product is supplied packed in a plastic bag. To avoid risk of suffocation, keep the bag out of the reach of children and infants.

## 4 Unpacking

The system is supplied in a box with the following contents:

- Combined reading machine and reading camera
- User Guide
- USB memory stick
- Power adaptor
- Hand control

Check first that the box has no external damage. In the event of damage, contact the delivery company and file a notification of damage.

Open the box and take out the MagniLink Vision; remove the packaging.

### 4.1 Packaging

The packaging consists of the following packaging material (LVI recommends that the packaging be kept in case you have to return the system or you need to move it):

- Packing material (EPP)
- Box (corrugated cardboard)
- Polyethylene (LDPE4)

For sorting and collection of packaging material, please call the packaging collection customer service department for information on the rules that apply in your municipality.


Note that the product and its accessories must be recycled as electronic waste. A worn-out battery is recycled as electronic waste.

## 5 Installation

Place your MagniLink Vision far enough in on the table in front of your seat, but at least 10 cm from the edge of the table. Remember that the reading table can be pulled out approximately 30 cm in three directions. MagniLink Vision has a low total weight. It is therefore very important to position the system in the right place on your table, particularly when you are using the reading table in its most extended position. An excessive load in this position can cause the system to tip over.

Connect the power cable to the wall socket and press the start button. The unit is now ready to be used. The screen can be tilted approximately 10 degrees backwards and lowered 8.5 cm from its highest position. Use the grips on both sides of the screen and move it to find the desired position. Release the brakes on the reading table and start using MagniLink Vision.

The following connection options can be found on the rear of the unit.

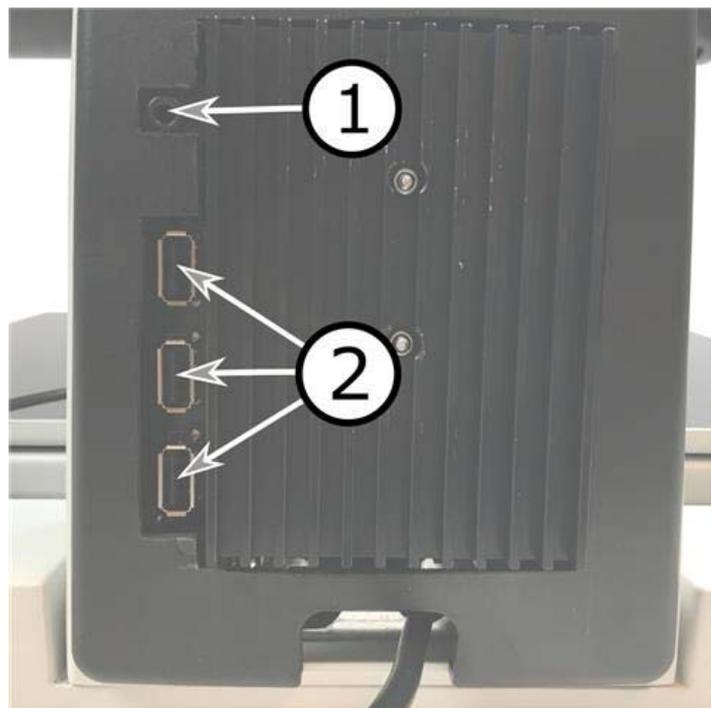


1. **Port for headphones**  
Here you can plug in headphones.
2. **USB ports**  
The unit has three USB ports. The hand control must be connected to one of these ports. If you wish to save or open a document, you can connect a USB memory stick.

## 6 Use

MagniLink Vision TTS is started from the panel. LVI's logo is displayed on the screen when the system is starting, and the picture from the CCTV camera is shown after ten seconds. After a further 1 minute and 25 seconds, the reading machine starts with the message "Reading machine ready". Using the hand control, it is now possible to switch between CCTV mode and OCR mode. When you are in CCTV mode, the panel is used to control the system. When you are in OCR mode, the hand control is used to control the system.

### 6.1 Panel

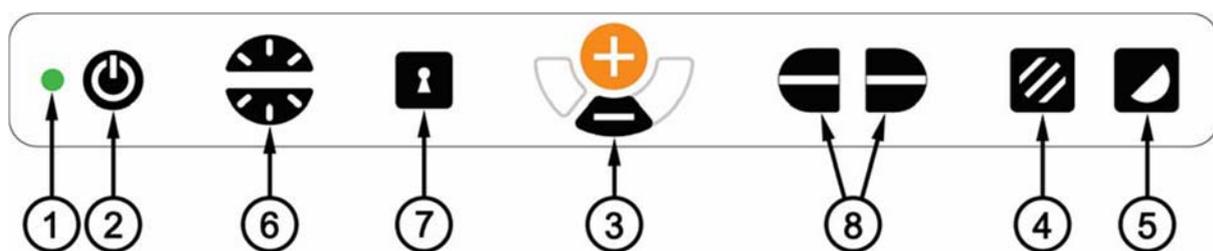

1. LED indicator  
Green light indicates that the unit is on. Flashing green light indicates start-up. A flashing yellow light means that autofocus is switched off. Red light indicates an error; refer to the section "Troubleshooting".
2. Power on/off  
Press the button to switch the unit on or off.
3. Magnification buttons  
To decrease the magnification, press the lower button (minus) and to increase the magnification, press the upper button (plus).
4. Colour/Gray scale picture mode  
Press button for a picture mode with natural colours or gray scale picture.

Light Adjustment (5 Push panel only)

Press and hold the button to temporarily adjust the light by pressing brightness control (6) up or down. This could be used,

for example, by light sensitive persons or when reading on glossy paper. The system will automatically switch between ROPC (Reading Optimized Picture Control) and DPC (Dynamic Picture Control) when the light reaches a certain level. There is no switching between natural colours and gray scale when the light adjustments are used.

### Light on/off

The light can be turned on and off by pressing the button for three seconds. The system will automatically switch between ROPC (Reading Optimized Picture Control) and DPC (Dynamic Picture Control).

5. High contrast mode with artificial colours  
Press the button to switch between different combinations of colours. Colour combinations available in the standard configuration:

Background/Text	Background/Text
White/Black	Red/Black
Black/White	Black/Red
Yellow/Black	Yellow/Blue
Black/Yellow	Blue/Yellow
Green/Black	White/Blue
Black/Green	Blue/White

6. Brightness control  
By pressing the buttons it is possible to adjust brightness. Press up to increase and down to decrease the brightness. When button picture mode (4) is pressed, the lamps intensity is adjusted instead. In this way, the best conditions for reading in different situations can be achieved.

7. **Autofocus on/off**  
Pressing the button locks the autofocus in the present position. This could be used, for example, when writing by hand without risk of disturbing the focusing system with the hand or the pen. A subsequent press of the button reactivates the autofocus system.
8. **Control for guiding line/curtain**  
Controls either the position of the guiding line or the size of the curtain. Press the right button to move the guiding line downwards. Continue to press the button and the guiding line switches from horizontal to vertical position and the guiding line will be moved from left to right. Further pressing the button and the guiding line turns into a horizontal curtain that reduces the visual area. Continue to press the button and the curtain becomes vertical.


The left button gives the opposite result.

## 6.2 Reading table



1. **Side way direction friction adjustment**  
Set the side way friction by moving this handle. With the brake in the left end position, the reading table is unlocked. To increase friction, push the side way brake to the right (at the far right, the brake clicks into locked mode).
2. **Depth way direction friction adjustment**  
Set the depth way friction by moving this handle. With the brake in the near end position, the reading table is unlocked. To increase friction, push the depth way brake towards the back. Push the brake in as far as possible to lock the reading table.

## 6.3 Profiles and settings

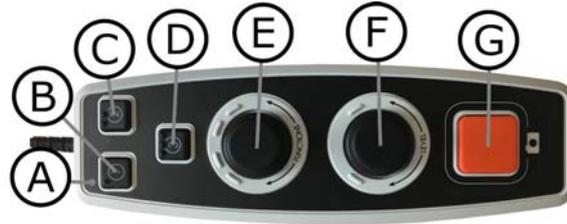

The unit can be used in simple profile or advanced profile. When the unit is in simple profile the LED lights green. When the unit is in advanced profile the LED lights blue. There is also a settings mode. When the unit is in settings mode it lights yellow.

To switch between simple and advanced profile, hold the control (E) pressed for five seconds.

To go into settings, hold the controls (E) and (F) pressed for three seconds.

## 6.4 Simple profile


### 6.4.1 Quick guide simple profile

**Rotary control**



**Push buttons**



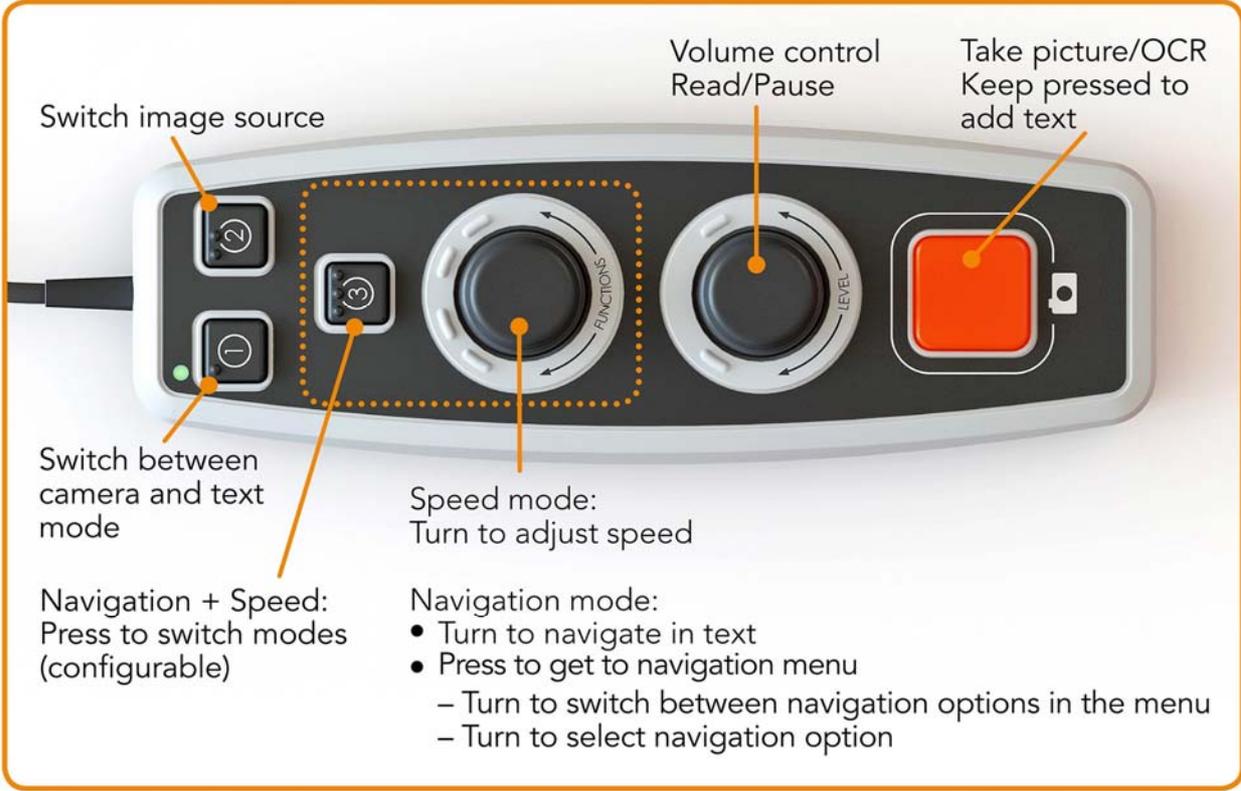

Switch image source

Switch between camera and text mode

Navigation + Speed: Press to switch modes (configurable)

Speed mode: Turn to adjust speed


Navigation mode:

- Turn to navigate in text
- Press to get to navigation menu
  - Turn to switch between navigation options in the menu
  - Turn to select navigation option

Volume control Read/Pause


Take picture/OCR Keep pressed to add text

**SETTINGS MENU**

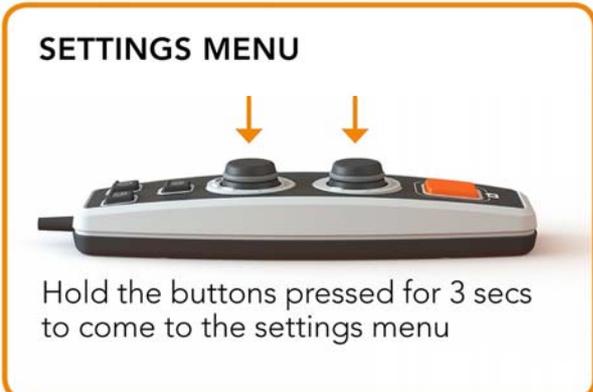


Hold the buttons pressed for 3 secs to come to the settings menu

**CHANGE PROFILE**

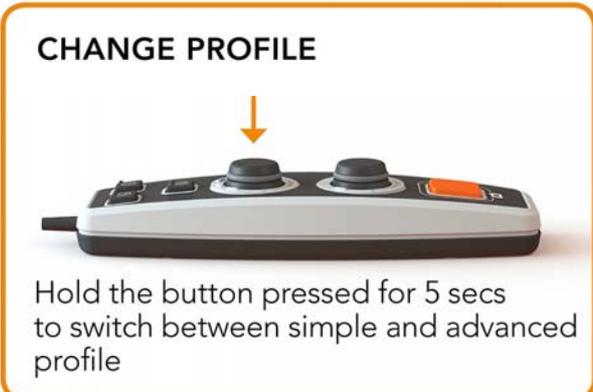

Hold the button pressed for 5 secs to switch between simple and advanced profile





## 6.4.2 Functions in simple profile

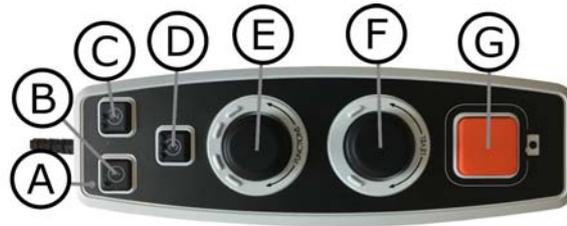


- A. **LED**  
In simple profile this LED lights green.
- B. **Switch between camera and text mode**  
This button switches between the OCR camera and the OCR processed text.
- C. **Switch image source**  
This button switches between the CCTV camera and OCR camera.
- D. **Quick button for navigation**  
When this button is pressed it switches to navigation mode. It is possible to configure this button so that it has another function.
- E. **Speed control**  
When you rotate the control, you adjust the speed of reading aloud.  
In navigation mode you can navigate in the text by turning the control. It is also possible to select navigation mode by pressing the control, turning it to the desired mode and then pressing it again.
- F. **Volume control**  
This is a combined push button and rotary control. When the control is pressed, you can start and pause the reading. When you rotate the control, you adjust the volume level.
- G. **OCR button**  
This button is used to take a picture, OCR process the text and read aloud the result. If you hold the button in for more than two seconds the text is added to a new page.

## 6.5 Advanced profile

### 6.5.1 Quick guide advanced profile

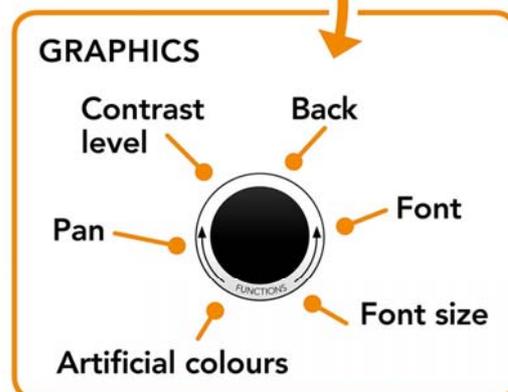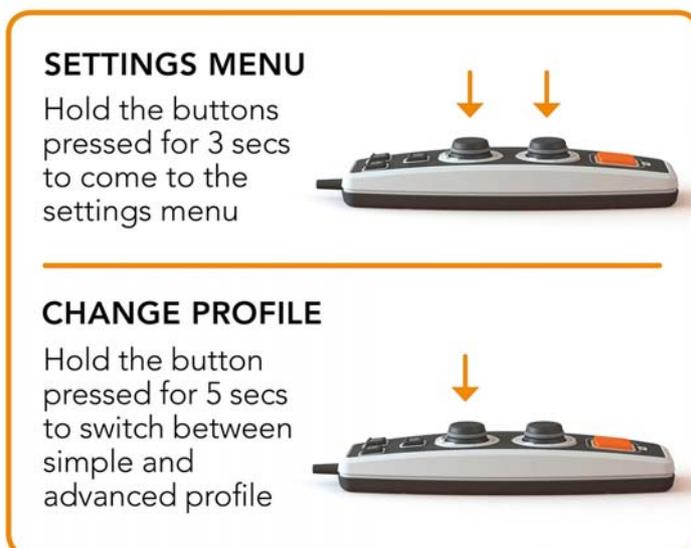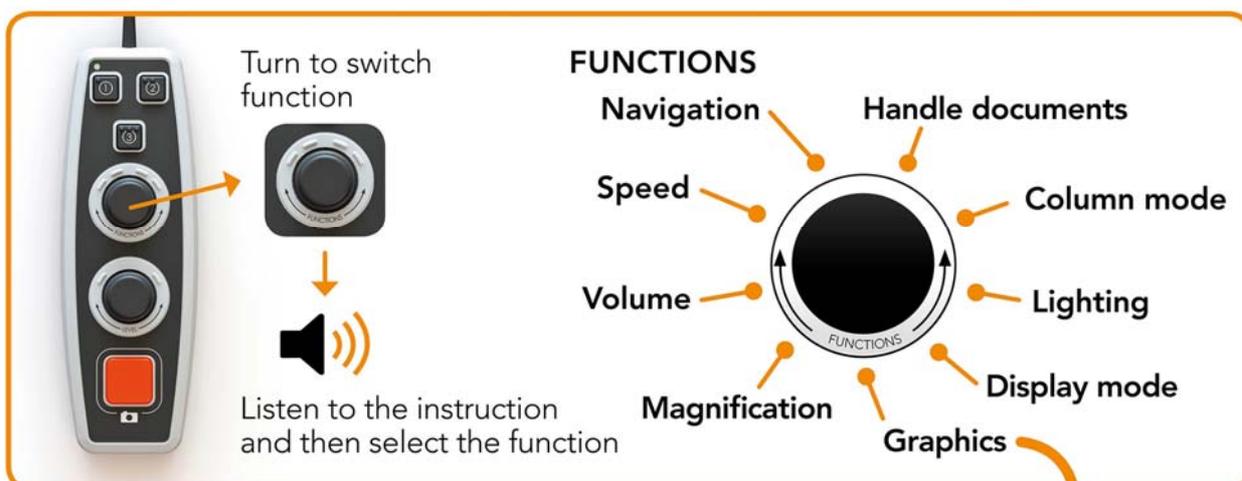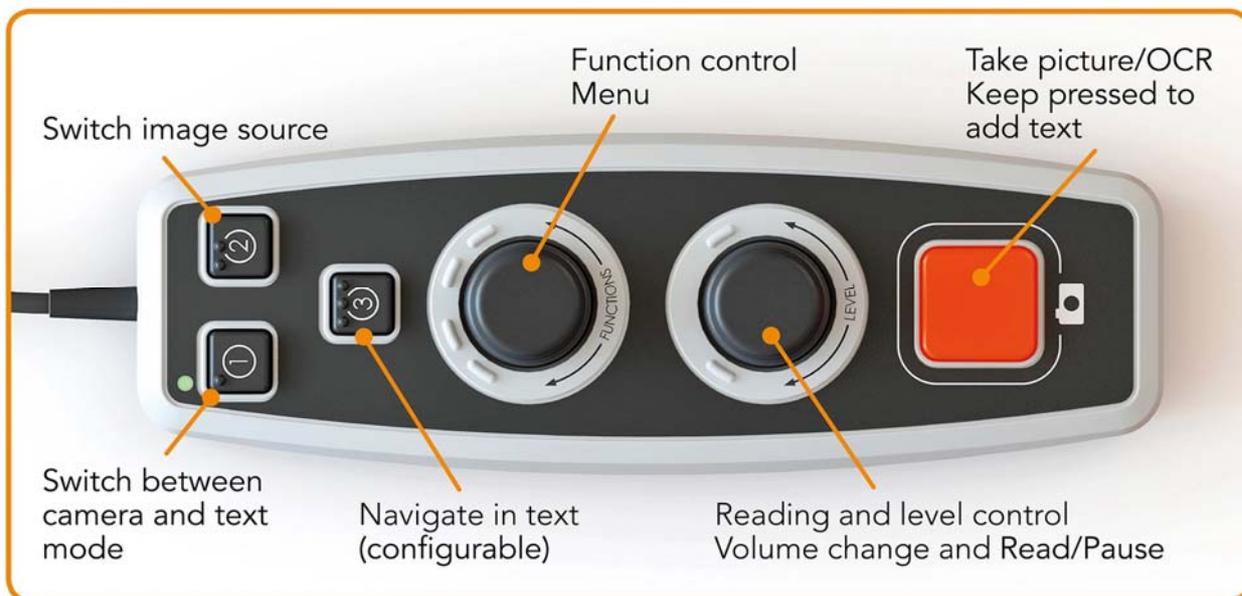

## 6.5.2 Functions in advanced profile

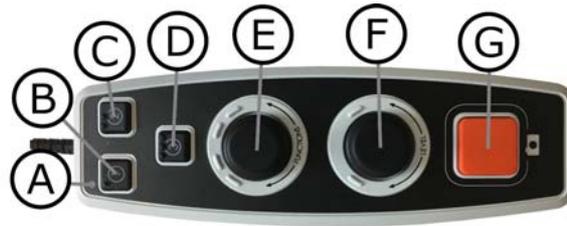

- A. **LED**  
In simple profile this LED lights blue.
- B. **Switch between camera and text mode**  
This button switches between the OCR camera and the OCR processed text.
- C. **Switch image source**  
This button switches between the CCTV camera and OCR camera.
- D. **Quick button for navigation**  
When you press this button you are brought to navigation, and it is now possible to navigate in the text by rotating the “Level control”.
- E. **Function control**  
This is a combined push button and rotary control. The control is rotated until the desired function is read aloud. Some functions have a sub-menu and pressing the control will enter this menu. Other functions have a setting that you can switch on or off by pressing the control.
- F. **Reading and Level control**  
This is a combined push button and rotary control. When the control is pressed, you can start and pause the reading. When you rotate the control, you adjust the relevant function.
- G. **OCR button**  
This button is used to take a picture, OCR process the text and read aloud the result. If you hold the button in for more than two seconds the text is added to a new page.

### 6.5.3 Describing functions in advanced profile

The unit's various functions and settings are controlled using the hand control. After a period of inactivity by the user, the unit returns to basic function. Below is a list of the unit's functions and descriptions of how they are performed.

#### Take a picture and start the reading

Press the OCR button(G) to take a picture to be OCR processed. When the OCR process is complete, the reading begins. If you hold the button in for more than two seconds the text is added to a new page.

#### Pause reading

Press the reading control(F) to pause the reading. Press it again to resume the reading.

#### Adjust the volume

Rotate the level control(F) to increase or decrease the volume.

#### Adjust the speech speed

Rotate the function control(E) until the unit says speed. Then rotate the level control(F) to increase or decrease the speech speed.

#### Save documents

Make sure that a USB memory stick is plugged into the unit. Rotate the function control (E) until the unit says "handle document" and then press the control. Rotate the function control again (E) until the unit says "save" and then press the control to save the document. The document is saved in the directory "lvi\_text".

#### Change file format

To change the format that you save to. Rotate the function control (E) until the unit says "handle document" and then press the control. Rotate the function control (E) again until the unit says "file format" and press the control. Rotate the control until the desired format is spoken and press the control to select it.

## Open document

Make sure that a USB memory stick is plugged into the unit. Rotate the function control (E) until the unit says "handle document" and then press the control. Rotate the function control (E) again until the unit says "open" and press the control. Turn again until the desired document is read aloud and press the control to open the document. The unit can open documents of the following file formats: .txt, .pdf, .rtf, .zip and image files: .png, .jpg, .tiff and .bmp. The documents must be in the directory "lvi\_text" or the root directory of the USB memory stick in order for the unit to find them.

## Delete documents

Make sure that a USB memory stick is plugged into the unit. Rotate the function control (E) until the unit says "handle document" and then press the control. Rotate the function control (E) again until the unit says "delete" and press the control. Turn the control until the file you wish to delete is read aloud and then press the control to delete the file.

## Navigate in the text

Rotate the function control(E) until the unit says "navigation". Then rotate the level control(F) to navigate forward or backwards in the text.

## Change navigation mode

Rotate the function control (E) until the unit says "navigation" and then press the control. Then choose the navigation mode using function control (E) and then press again. It is now possible to navigate in the text using the level control (F). There are five different navigation modes: "page", "block", "sentence", "word by word" and "letter by letter". If the display mode is set to silent mode, there is also the navigation mode "view".

## Change column mode

To change column mode, rotate the function control(E) until the unit says "column mode" and then press the control. Normally the column mode is on, but it is necessary to turn it off when you are processing text that you do not want to have read aloud in column form.

### **Change display mode**

To change display mode, rotate the function control (E) until the unit says "display mode" and then press the control. Then select display mode by rotating the function control (E). When you are satisfied with your selection, press the control (E) again. There are six display modes with speech: "word", "line", "page", "picture", "picture and word" and "picture and line". There are also two silent modes. One where the text is moved horizontally on one line and one where the text is moved vertically in a column.

### **Change magnification**

Rotate the function control (E) until the unit says "magnification". Then rotate the level control (F) to increase or decrease the magnification.

### **Change font**

Rotate the function control (E) until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "font" and press the control. Rotate the control until the desired font is read aloud and press the control to select this.

### **Change font size**

Rotate the function control (E) until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "font size". It is now possible to change the font size by rotating the level control (F).

### **Change artificial colours**

Rotate the function control (E) again until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "artificial colours" and press the control. Rotate the control until the desired colours are read aloud and press the control to select these.

### **Change contrast level**

Then rotate the function control (E) until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "contrast level". It is now possible to change the contrast level by rotating the level control (F). This setting only has an effect if any artificial colours are selected.

## 6.6 Settings

### 6.6.1 Quick guide settings

**SETTINGS MENU**




Hold the buttons pressed for 3 secs to come to the settings menu



Turn to switch function

Listen to the instruction and then select the function

**FUNCTIONS**

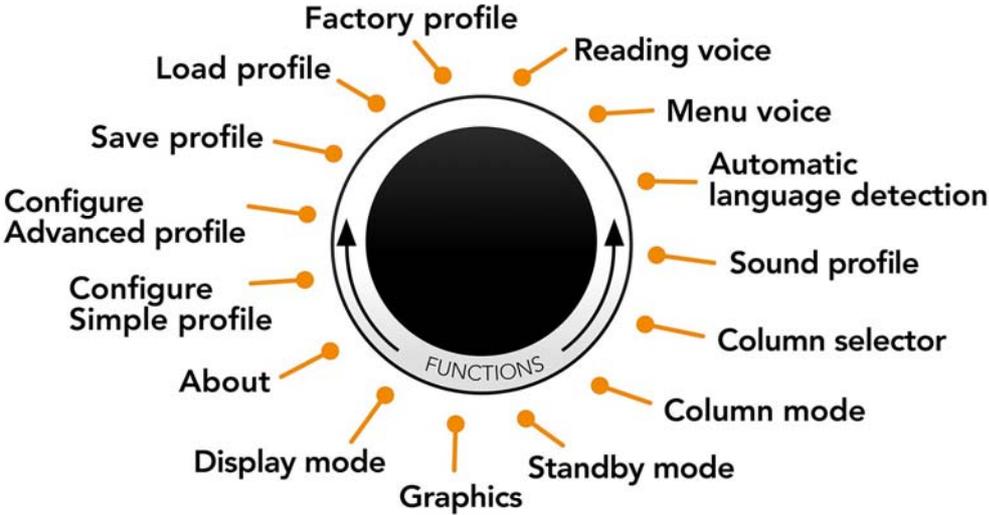

Factory profile

Load profile

Save profile

Configure Advanced profile

Configure Simple profile

About

Display mode

Graphics

Standby mode

Reading voice

Menu voice


Automatic language detection

Sound profile

Column selector

Column mode

**GRAPHICS**

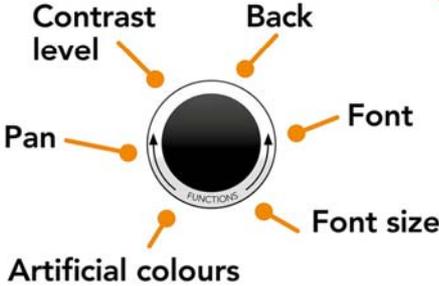

Contrast level

Back


Font

Font size

Artificial colours

Pan

## 6.6.2 Functions in settings

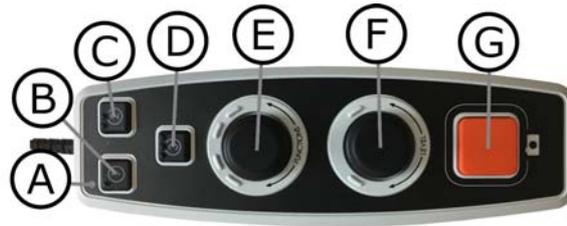

- A. **LED**  
When the unit is in the settings menu it lights yellow.
- B. **Switch between camera and text mode**  
This button switches between the OCR camera and the OCR processed text.
- C. **Switch image source**  
This button switches between the CCTV camera and OCR camera.
- D. **No function**
- E. **Setting control**  
The control is rotated until the desired setting is read aloud. Some settings have a sub-menu and pressing the control will enter this menu. Other settings have a setting that you can switch on or off by pressing the control.
- F. **Level control**  
Used to set font size and lighting.
- G. **Exit button**  
This button is used to exit the settings menu.

## 6.6.3 Describing the settings options

### Change reading voice

Rotate the function control (E) again until the unit says "reading voice" and press the control. Rotate the function control again (E) until the desired voice is spoken and then press the control to select the voice.

### Change menu voice and language setting

Rotate the function control (E) again until the unit says "menu voice" and press the control. Rotate the function control again (E) until the desired voice is spoken and then press the control to select the voice.

### Change automatic language detection

Then rotate the function control (E) until the unit says "automatic language detection" and press the control to change the setting.

### **Change sound profile**

Then rotate the function control (E) until the unit says "sound profile" and press the control. Rotate the function control again (E) until the desired sound profile is read aloud and then press the control to select. There are three different sound profiles: "normal", "bass" and "treble".

### **Change column selector**

To change column selector, rotate the function control(E) until the unit says "column selector". It is now possible to adjust the column selector by pressing the control. There are three options: vertical, horizontal or off. When the column selector is on, only those columns that are intersected by the column selector are OCR-processed.

### **Change column mode**

To change column mode, rotate the function control(E) until the unit says "column mode" and then press the control. Normally the column mode is on, but it is necessary to turn it off when you are processing text that you do not want to have read aloud in column form.

### **Change standby mode**

To set the time after which the unit will switch to standby mode. Rotate the function control (E) again until the unit says "standby mode" and press the control. Rotate the function control again (E) until the desired time is read aloud and then press the control to select it.

### **Change font**

Rotate the function control (E) until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "font" and press the control. Rotate the control until the desired font is read aloud and press the control to select this.

### **Change font size**

Rotate the function control (E) until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "font size". It is now possible to change the font size by rotating the level control (F).

### **Change artificial colours**

Rotate the function control (E) again until the unit says "graphics" and press the control. Rotate the function control (E) again until the unit says "artificial colours" and press the control. Rotate the control until the desired colours are read aloud and press the control to select these.

### **Change contrast level**

Then rotate the function control (E) until the unit says “graphics” and press the control. Rotate the function control (E) again until the unit says “contrast level”. It is now possible to change the contrast level by rotating the level control (F). This setting only has an effect if any artificial colours are selected.

### **Change display mode**

To change display mode, rotate the function control (E) until the unit says “display mode” and then press the control. Then select display mode by rotating the function control (E). When you are satisfied with your selection, press the control (E) again. There are six display modes with speech: “word”, “line”, “page”, “picture”, “picture and word” and “picture and line”. There are also two silent modes. One where the text is moved horizontally on one line and one where the text is moved vertically in a column.

### **Get information about the unit's serial number**

Rotate the function control (E) until the unit says "about" and then press the control. Rotate the function control again (E) until the unit says "serial number" and press the control. The unit's serial number will now be read aloud.

### **Get information on the version of the software**

Rotate the function control (E) until the unit says "about" and then press the control. Rotate the function control again (E) until the unit says "version" and press the control. The unit will now read aloud information about the software version.

### **Update license**

Updating the unit's license requires a USB memory stick with a license file to be plugged into the unit. Rotate the function control (E) until the unit says "about" and then press the control. Rotate the function control again (E) until the unit says "update license" and press the control. The unit will then notify you if the licence update was successful.

### **Update software**

Updating the unit's software requires a USB memory stick with new software to be plugged into the unit. Rotate the function control (E) until the unit says "about" and then press the control. Rotate the function control again (E) until the unit says "update software" and press the control. When the update is complete, the unit will notify you of this.

### **Save profile**

Make sure that a USB memory stick is plugged into the unit. Rotate the function control (E) until the unit says "save profile" and then press the control.

### **Load profile**

Make sure that a USB memory stick is plugged into the unit. Rotate the function control (E) until the unit says "load profile" and then press the control. Rotate the function control again (E) until the desired profile is read aloud and then press the control to load it. The profiles must be in the root directory of the USB memory stick in order for the unit to find them.

### **Factory profile**

Rotate the function control (E) until the unit says "factory profile" and then press the control.

### **Select basic function**

To select which menu the unit will return to after a period of inactivity. Then rotate the function control (E) until the unit says "configure advanced profile". Rotate the function control again (E) until the unit says "basic function" and press the control. Rotate the function control again (E) until the desired option is read aloud and then press the control. It is possible to choose between volume, speed, magnification and navigation.

### **Select quick function**

To select quick function move quick button (D). Then rotate the function control (E) until the unit says "configure advanced profile" and press the control. Rotate the function control again (E) until the unit says "quick function" and press the control. Rotate the function control again (E) until the desired option is read aloud and then press the control. It is possible to select between display mode, column selector, column mode, navigation, automatic language identification and speaking voice.

## Remove functions

To remove functions. Then rotate the function control (**E**) until the unit says "configure advanced profile" and press the control. Rotate the function control again (**E**) until the unit says "functions" and press the control. It is now possible to navigate and remove functions using the function control. Press the control to remove or restore the selected function. When you have selected the functions you wish to be available, navigate with the function control (**E**) until the unit says "back" and press the control.

## Configure simple profile

It is possible to configure the quick button (**D**). Then rotate the function control (**E**) until the unit says "configure simple profile" and press the control. Rotate the function control again (**E**) until the desired option is read aloud and then press the control. It is possible to select between display mode, column selector, column mode, navigation, automatic language identification and speaking voice and no function.

## 6.7 Quick guide computer mouse

### IN CAMERA MODE

By pressing the left mouse button, you take a picture and switch to OCR mode.



By pressing the right mouse button you switch to OCR mode (if the picture is already OCR processed).

By rolling the scroll wheel you can zoom in on the image.

### IN OCR MODE

Press the left mouse button and mark the text that you want OCR processed.

By holding the scroll wheel and then moving the mouse cursor you can pan in the picture (if it is enlarged).



By pressing the right mouse button you switch to camera mode.

By scrolling with the scroll wheel you can zoom in the image and change the font size:

- If the mouse cursor is in the lower section of the monitor the font size is changed.
- If it is in the upper section you zoom in the picture.

## 6.8 Computer mouse

Certain functionality can be controlled with the mouse.

In camera mode.

- By pressing the left mouse button, you take a picture and switch to text mode.
- Rolling the scroll wheel can zoom in on the image.
- By pressing the right mouse button you switch to text mode (If a picture has been taken).

In text mode

- By pressing the left mouse button and marking the text that you want OCR processed.
- By scrolling with the scroll wheel you can zoom in the image and change the font size. If the mouse cursor is in the lower section of the monitor the font size is changed. If it is in the upper section you zoom in the picture.
- By holding down the scroll wheel and then moving the mouse cursor you can pan in the picture (If it is enlarged).
- By pressing the right mouse button you switch to camera mode.

## 6.9 Extra voices on a USB stick

It can be possible to use voices that are on an external USB drive. To enable the voices, connect the USB stick to one of the unit's USB ports and wait until the unit says "voices are ready to be used". It is then possible to go into the menu and change voices and language settings to one of the new voices.

## 6.10 Quality of OCR conversion

In order to optimize the results of the OCR conversion, you should bear in mind the following.

- Make sure that the text is large enough. If the text is too small, the OCR results will be bad. At least 12pt is recommended.
- Avoid glossy paper as this results in reflections in the image. This impairs the results of the OCR conversion. If you intend to use glossy paper, we recommend that the light on the unit is switched off.
- Certain fonts can be difficult for the OCR engine to interpret.
- It is important that the text contrast is sufficient.
- It is important that the chosen spoken language is the same as that of the text.
- The paper should be rotated as little as possible.

## 7 Ergonomics

It is important that the workplace around your system is comfortable, and that you sit correctly. This is so that you can use it for as long as possible without putting undue strain and stress on your neck and back.

You should bear the following in mind.

- Your chair should be positioned close to the table on which the equipment is located, and you should not sit in the chair for too long. We recommend a good office chair with adjustable height.
- The seat of the chair should be at a height that does not cause your legs to become numb; legs should be at an angle of 90 degrees and you should rest your feet on the floor.
- The reading table should be at a height at which you can rest your lower arm horizontally on it. It is important that the table is neither too high nor too low for you. The ideal solution is if the screen is slightly below eye level and in as straight a line in front of you as possible.
- Shield out daylight. Lights in the ceiling, table lamps and light from windows can cause reflections on the monitor. For this reason, avoid placing lights where they shine directly at the screen or at yourself.
- If you use the device for long periods of time, there is a risk of repeated movements causing you pain in your neck, shoulders and back. Your eyes can also get tired from looking at a screen for too long.

## 8 Troubleshooting

No image is displayed after you start the system:

- Check that the power is on.
- Check that all connectors are properly connected as explained in the 'Installation' section.

The LED to the left of the start button shines red:

- Something is wrong and MagniLink Vision cannot be used. Press the start button once and MagniLink Vision will make three attempts to restart. If the system still fails to start, the LED will shine red again. Disconnect the power cable and contact service personnel.

The LED flashes red:

- The incorrect voltage is being supplied from the mains adapter. If the fault persists, disconnect the power cable and contact service personnel.

If the unit gives the voice response "No text in the document" after an OCR conversion.

- Check that the document is not rotated through more than 15 degrees.

Never remove any panels or the casing in order to attempt to repair the system yourself. In the event of problems, contact your local dealer.

## 9 Technical information

Magnification	2,0 - 70x
Monitor size	23"
Resolution	1920 x 1080
Power consumption	Max 67 W
Dimensions in use	22 x 18.5 x 22.4 in (W x D x H)
Weight	29.7 lbs
Refresh rate CCTV camera	60 Hz
Refresh rate OCR camera	7 Hz
Luminance	300 cd/m <sup>2</sup>
Contrast	1000:1
Focusing distance	0-23.6 in (with max. magnification)
Adjustable working height	43.3 in – 76.8 in
Light source	LED
Mains voltage	100-240 V, 50-60 Hz
Reading table dimensions	16.5 x 11.8 in (W x D)
Acceptable temperature ranges	Transport and storage: -4°F – +140°F Use: +41°C – +77°C
Relative humidity	<80%



**Disposal of old electrical and electronic equipment (applies for EU countries and all other European countries that have specific collection systems)**

If the product or its packaging has this symbol on it, it is not to be handled as regular domestic waste. It should instead be deposited at a suitable collection centre for the recycling of electrical and electronic equipment. By ensuring that this product is properly taken care of, you are helping to prevent the negative impact on health and the environment that might otherwise result from inappropriate disposal of this product. Recycling helps in the management of our natural resources. For more detailed information regarding the recycling of this product, you can contact your municipality, your waste management company or the store where you purchased the product.

## 10 Declaration of Conformity

### DECLARATION OF CONFORMITY

Manufacturers name: *LVI Low Vision International AB*

Manufacturers Address: *Verkstadsgatan 5  
S-352 46 Växjö  
Sweden*

Web: *http://www.lviglobal.se*

SRN (Single Registration Number): *SE-MF-000004178*

Basic UDI-DI (GMN): *7330724MLVEJ*

Brand name or trademark: *MagniLink*

Product code: *MLV-HD-E, MLV-FHD-E*

Classification: *Class I*

Type of equipment: *Reading Aid*

Conformity assessment route: *LVI Low Vision International AB uses the following procedures for the CE-labeling of their products according to the Regulation MDR 2017/745:  
Class [1]: EC conformity declaration according to Annex I + Annex II + Annex VII*

This declaration of conformity is issued under the sole responsibility of LVI Low Vision International AB. We hereby declare that the medical device(s) specified above meet the provision of the Regulation (EU) MDR 2017/745 for medical devices. This declaration is supported by the Quality System approval to ISO 9001 issued by Intertek. All supporting documentation is retained at the premises of the manufacturer.

Manufacturer       Manufacturer's authorized representative

Date

20-05-2021

Signature



Position

Managing Director

Clarification

Henrik Blomdahl

## Alphabetical index

About LVI .....	5
Advanced profile .....	17
Cleaning .....	7
Computer mouse .....	29
Declaration of Conformity .....	35
Describing functions in advanced profile .....	19
Describing the settings options .....	23
Ergonomics .....	31
Extra voices on a USB stick .....	29
Functions in advanced profile .....	18
Functions in settings .....	23
Functions in simple profile .....	16
Information .....	7
Installation .....	10
Introducing MagniLink Vision TTS .....	6
Packaging .....	9
Panel .....	11
Profiles and settings .....	14
Quality of OCR conversion .....	30
Quick guide advanced profile .....	17
Quick guide computer mouse .....	28
Quick guide settings .....	22
Quick guide simple profile .....	15
Reading table .....	13
Reuse, installation and maintenance .....	7
Safety information .....	7
Settings .....	22
Simple profile .....	15
Technical information .....	33
Troubleshooting .....	32
Unpacking .....	9
Use .....	11