v1.0, 2017-03, V3C QSG English/Canadian French

**BT:** 2.4GHz ISM Band 2.402-2.480GHz **Output Power:** +4dBm

**Model No:** B1002-S **NFMB:** 10.5MHz

**Left FCC ID:** EOA-2AF5TB1002L **Right FCC ID:** EOA-2AF5TB1002R

**Left IC:** 6903A-2AF5TB1002L **Right IC:** 6903A-2AF5TB1002R

**Starkey Hearing Technologies**, Minneapolis, MN, USA & **Bragi GmbH**, Munich, Germany

Starkey is a trademark of Starkey Laboratories, Inc., registered in the U.S. and other countries.

All rights reserved. Patents pending. BRAGI, The Dash, "The World's First Hearable", "Listen.

Track. Communicate.", and the B logo are trademarks of Bragi GmbH in the U.S. or one or more other countries. The absence of a product or service name or logo from this list does not constitute a waiver of trademark or other intellectual property rights concerning that name or logo.

# The Dash<sup>PRO</sup>

Tailored by Starkey<sup>®</sup>  
Hearing Technologies

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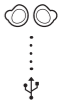
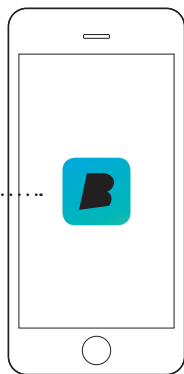
## Bragi App

Download the free Bragi App to set up  
and remote-control your Dash.

Unlock all of its advanced features now!

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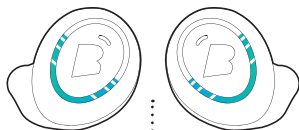
# Charge Charger

Before using The Dash, charge it for at least 1 hour. This will also activate your Dash.

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plein full   
haut high   
moyen medium   
bas low 

## Shake.

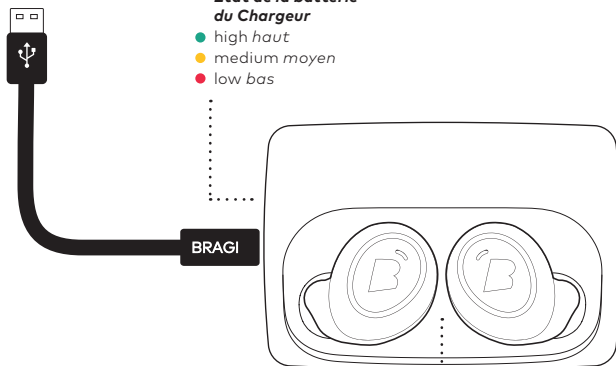
See the current battery status by shaking your Dash.

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 high haut  
 medium moyen  
 low bas

**The Charger will power  
The Dash up to 5 times  
while on-the-go.**

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With The Dash in The Charger, connect to your computer or a power supply via USB.

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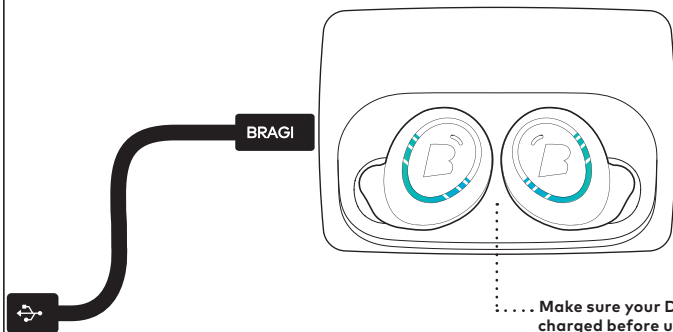
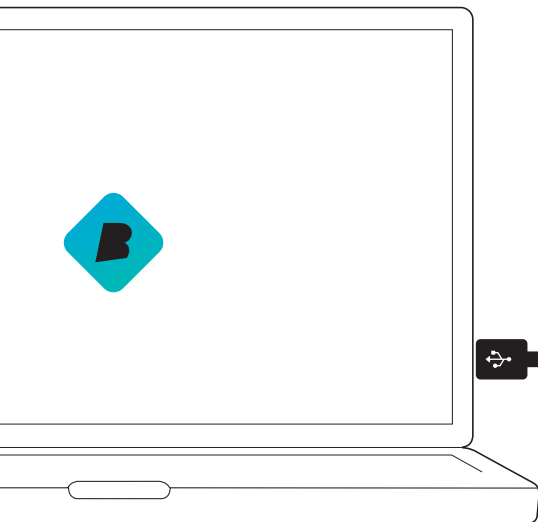
Your Dash learns new skills when updated to the newest **Bragi OS**.  
Download from **update.bragi.com**

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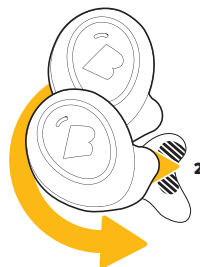
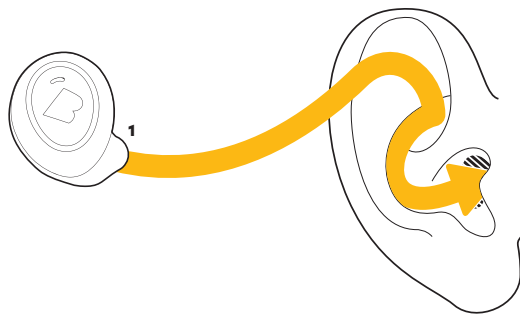
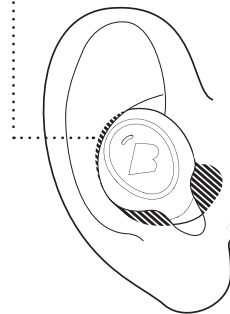
[bragi.com/support](https://bragi.com/support)

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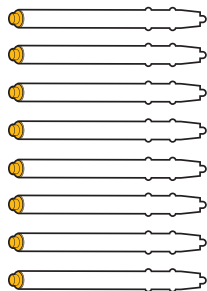
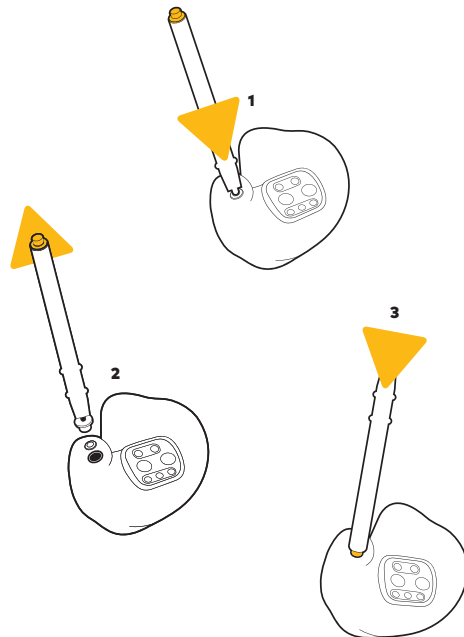
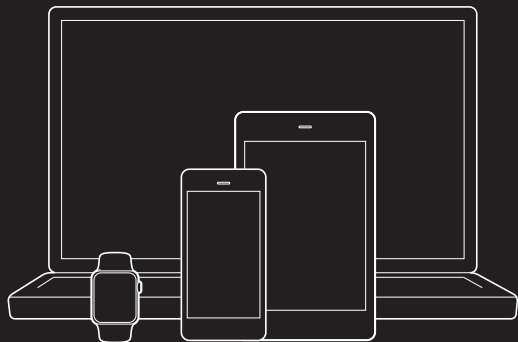
**And now ...**  
***Et maintenant...***

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
## First connect audio

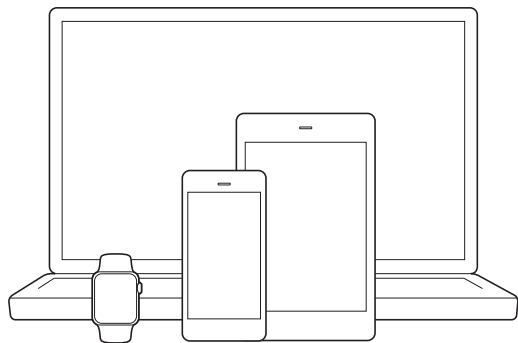Insert The Dash into your ear and search for **The Dash** in the Bluetooth® menu of your device.


- ! **If your Dash does not show up in the list, do a Long Hold (5sec) on the Right Dash. You can find video tutorials on [bragi.com/support](http://bragi.com/support)**

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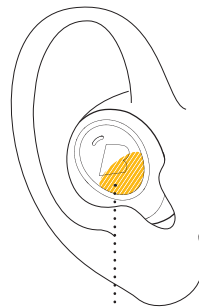
## Then connect sensors

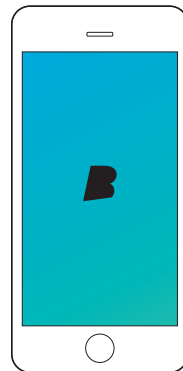To connect The Dash sensors to your device, open the Bragi App and select your Dash in the device list. Make sure to connect The Dash audio system first before connecting the sensors.

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# EarTouch

Control your Dash with tap, hold and swipe gestures using the EarTouch Interface.

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## **Starkey Limited Customization Refit**